

Bravo Checking Printer Paper Feed

ATTACHMENT F - CHECKING THE PRINTER PAPER FEEDING



Before servicing, switch off the equipment and unplug the power supply cable from the mains socket.

- Open the service compartment.
 - Switch on the sterilizer.
 - Push the key on the right of the printer's front panel, and check that the paper advances from the slot.
-
1. If so, run a sterilization cycle and check the print report at the end of the cycle.
 2. Otherwise, check:
 - a. the fuse on the printer power supply board,
 - b. the correct connection on the electronic boards,
 - c. replace the printer unit, if necessary

