


Bravo Distilled Water Reservoir Cleaning

DAILY	Clean the door gasket Clean external surfaces
WEEKLY	Clean the sterilization chamber and relative accessories Disinfect external surfaces
MONTHLY	Clean the internal (and external - if installed) distilled water tank Safety valve maintenance Clean (or replace) the drain filter
EVERY 3-6 MONTHS (depending on frequency of use)	Replace bacteriological filter
ANNUAL or every 1000 cycles	Replace the door gasket
EVERY 3 YEARS or 3000 CYCLES (by approved personnel only)	Recommended complete maintenance and calibration of the sterilizer

Cleaning the internal tank Bravo

Clean internal distilled water tank

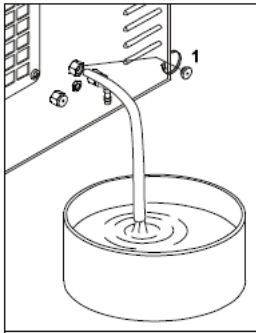

1. Arrange an empty container on the floor near the sterilizer and insert the free end of a tube.
2. Unscrew the plug (1) from the rear draining point and plug in the other end of the tube.
3. Wait until the internal tank is completely drained and close the draining point with the plug.
4. Prepare 4 litres / 1.06 US gal of distilled water mixed with 10% of pure alcohol and fill the supplied standard container
5. Fill the internal tank completely with this solution (see Chapter 5 – Instructions for Use **Filling distilled water** for the procedure) and allow the solution to sit for 30 minutes.


WARNING



DO NOT RUN ANY CYCLE DURING THIS PERIOD.

6. Now drain the internal tank and discard the solution. Close the draining point with the plug.
7. Run one empty cycle of your choice

Note: To make the cleaning process easier, purchase a Bravo Water Reservoir Drain Kit #01-112320S.

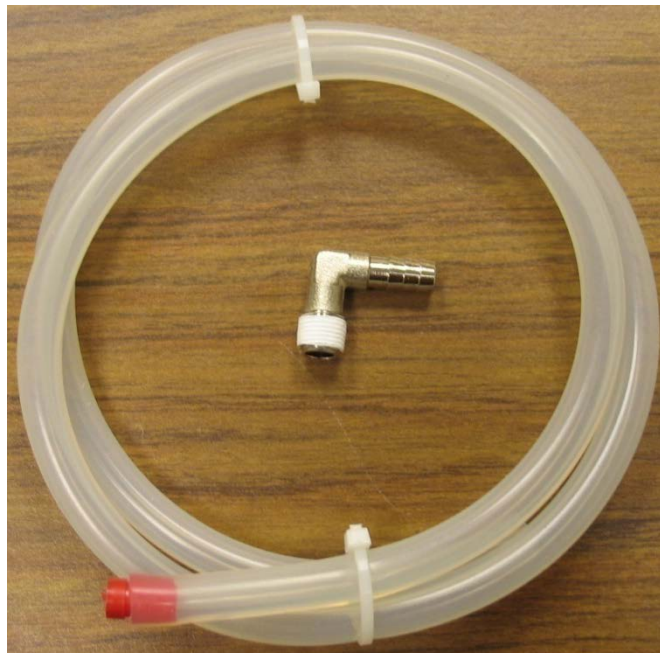

Cleaning the internal tank Bravo 2

For the occasional disinfection of the external surfaces, you can use either denatured alcohol or detergents with a small percentage of sodium hypochlorite (or equivalent).

1. Arrange an empty container on the floor near the sterilizer and insert the free end of the tube.
2. Insert the other end of tube in the quick-coupling marked "Service" positioned on the front as shown in the figure below.
3. Allow the tank to empty completely, and then disconnect the tube.
4. Prepare 4 liters/1.06 US Gal of distilled water and 10 % of pure alcohol, such as isopropyl and then pour it into the distilled water tank following the procedure indicated in the chapter "Loading Distilled Water" until the maximum level has been reached.
5. Let the solution sit for 30 minutes.


WARNING

IN THE MEANTIME, DO NOT CARRY OUT ANY STERILIZATION CYCLES.






6. Repeat step 2 and **completely** empty the internal tank again (as in point 2).
7. Run one empty (no load) cycle of your choice.

Bravo Distilled Water Reservoir Drain Kit Installation Instructions Part #01-112320S


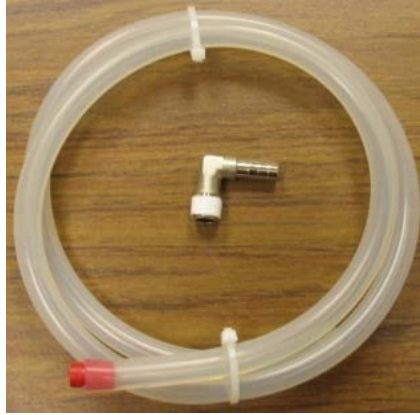


Turn the main power switch OFF and unplug the unit. Go to the rear of the Bravo with an empty container and remove the plug to the clean water drain port. **Note: As soon as the plug is removed water will come out of the port.**

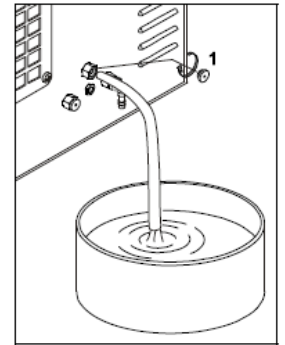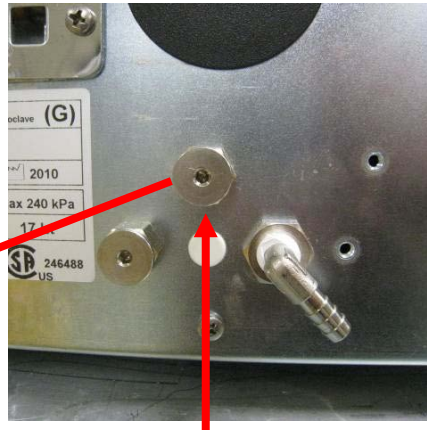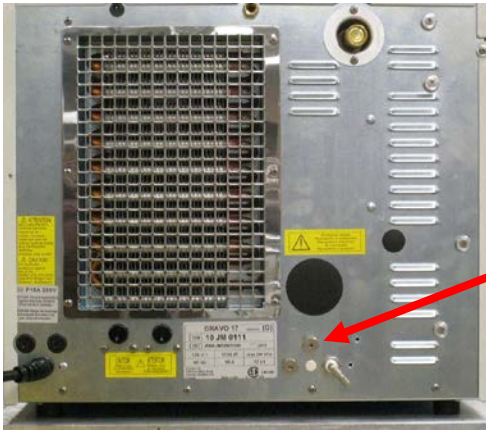

**Distilled Water
Drain Port**

Once the distilled water tank is drained install the elbow fitting into the distilled water drain port.

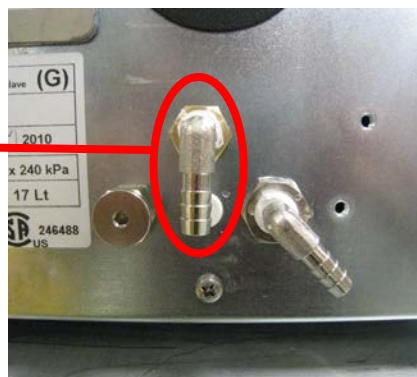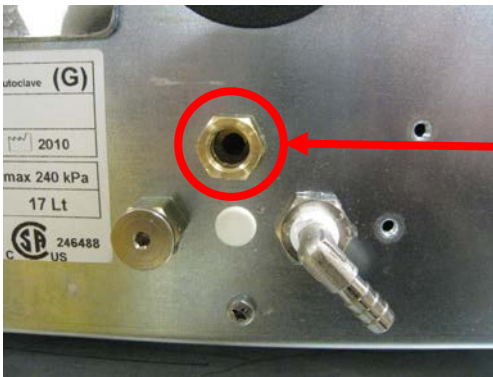

Connect the drain tubing to the elbow fitting using a tie wrap and install the plastic drain tube holder underneath the rear of the Bravo, below the elbow fitting. Feed the drain tubing through the holder. Mount the second drain tube holder underneath the front of the Bravo below the H2O drain sticker. Make sure the orange plug is installed into the end of the drain tubing.

