

# Bravo Printer Kit Installation

## ATTACHMENT D - PRINTER KIT INSTALLATION

**A9BP5430000**  
**95530152**


**Printer Kit Gam**  
**Printer Kit Troll**



**Before servicing, switch off the equipment and unplug the power supply cable from the mains socket.**

### Style 1

1. Remove the frame cover (see card **Gr7-1**).
2. Mount on the electronic board support the printer's power supply board (1) provided with kit
3. Cut the clamp on the transformer wiring and connect the free wiring to the AC input connector.
4. Place the printer unit (2) in the the service compartment and fix it back by means of the screw (3)
5. Connect the printer cable (4) between the output connector of the power supply board and the power supply input connector of the printer unit; arrange the cable path on the top of the electronic board support and fasten it by nylon clamps.
6. Connect the signal cable (5) between the CPU board and printer unit connectors.
7. Enter the SETUP mode, select ADVANCED menu, PRINT OPTIONS, PRINTER, and the INTERNAL option.
8. Exit the SETUP mode.
9. Run a sterilization cycle; at the end of the cycle check the correct print report.

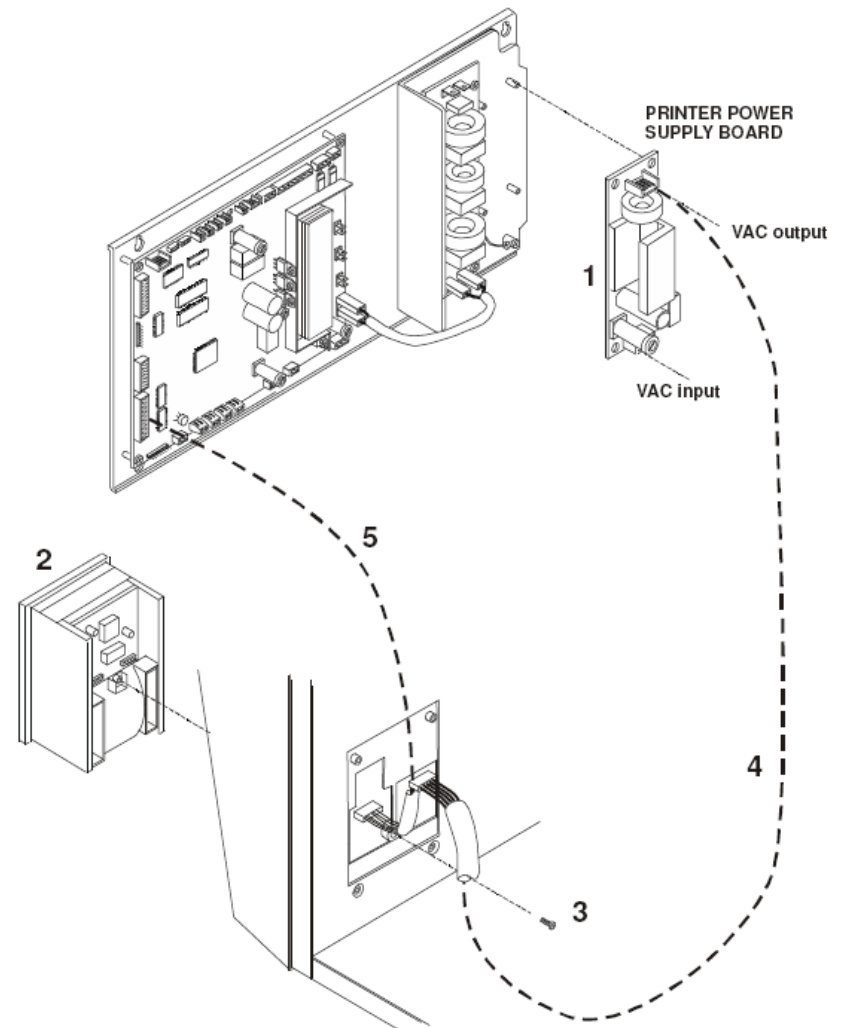

## Style 2

1. Remove the frame cover (see card **Gr7-1**).
2. Replace the old printer power supply board with the one of the printer kit
3. Fit the printer unit (1) in the kit's printer frame (2).
4. Fix the printer support plate (3) on the service compartment by means of four screws (4).
5. Now fix the framed printer unit on the support plate by means of three screws (5).
6. Restore the the printer cable connections.
7. Enter the SETUP mode, select ADVANCED menu, PRINT OPTIONS, PRINTER, and the INTERNAL option.
8. Exit the SETUP mode.
9. Run a sterilization cycle; at the end of the cycle check the correct print report.

