


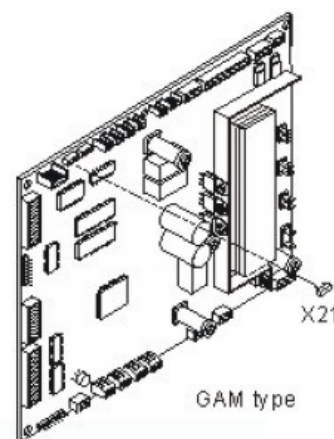
# Bravo Resetting Printer Queue

## ATTACHMENT E - RESETTING THE PRINT QUEUE




Before servicing, switch off the equipment and unplug the power supply cable from the mains socket.

1. Remove the frame cover (see card **Gr7-1**).
2. Remove the jumper X21 of the CPU board.
3. Push the key (hidden) of the control panel, switch-on the sterilizer. Release the key when the message "*LOCKING DEVICE*" appears on the LCD display.
4. Push the key  to exit this status. The auto-test will start.
5. When completed the auto-test, switch off the sterilizer, restore the jumper X21 and switch on again the unit.
6. Enter the SETUP mode. Select ADVANCED menu, PRINT OPTIONS, REPORT, PRINT LAST and the NORMAL PRINT option.
7. Exit the SETUP mode, and check that the printer resumes to operate correctly.



GAM type

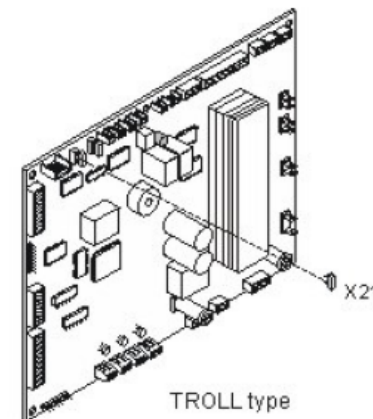

TROLL type



Style 2