

C51W Fault Codes

E1-Error light flashes once pauses flash once pauses etc.

The chamber does not get hot enough.

1. Check fuse F3 2A 250V
2. Check hot water supply.
3. Check the lower recirculation pump is running.
4. Check the heater assembly.

E2-Error light flashes twice pause flashes twice pauses etc.

The chamber does not fill with water.

1. Check water inlet valves and filters.
2. Check chamber full pressure switch.

E3-Error light flashes three times pauses flashes three times pauses etc.

The temperature is outside the expected range.

1. Check thermostat on the heater assembly.

E4-Error light flashes four times pauses flashes four times pauses etc.

The chamber does not drain.

1. Check drain hose
2. Check drain pump