

## Checklist for Equipment Rep before Install Technician Arrives to install a Statim G4

1. Cat 5 Ethernet Cable run to Sterilization Area and connected to network
2. Contact information (fill out attached contact sheet for install technician)
3. Gmail or other email account set up with address and password for outgoing messages (set up account with username and password for cycle emails)
4. Email addresses for those receiving cycle data
5. Individual names and 4 digit password for optional password protected cycles (total of 4, optional)
6. Timing of Maintenance notifications if applicable (# of cycles or time; optional)
7. Doctor Password if password protection is desired (default password is SciCan...make sure that new password is written down and easily remembered by the doctor or office staff if changed)
8. Location on network to backup files (optional)
9. Dealer Number for Marketing (optional)
10. Location of Office Logo on system (optional)

### Other Considerations:

1. Complete Warranty information at time of setup on Contacts screen. Fields will be populated with information input into the Contact Screen
2. STATFlow checklist completed if STATFlow being installed
3. Determination of which rack will be used if Statim 5000 G4 is purchased for setup
4. Access to network computer for install technician to setup menus on G4