

Hydrim C61 G4 AC Power Inlet Removing & Reinstalling Part #01-110505S



RO valve (if fitted)

fuses

screws

Figure 48

Removing & Reinstalling AC Power Inlet (Line Filter)

Disconnect the detergent bag and remove bag and drawer.

Run a shipping cycle to drain the unit, disconnect power and flip unit onto its back.

Remove bottom cover.

Remove wires from power inlet.

Unscrew mounting screws to remove AC power inlet (Figure 48)

To reinstall, reverse instructions.

To reinstall the bottom cover, slide the cover back into position, replace the screws and pull the rubber standoffs back through the cover. (**NOTE:** if the rubber standoffs are not pulled through the cover, the dryer inlet duct will not function correctly.)