

## Hydrim C61 G4 Bottom Cover Removing & Reinstalling Part #01-113296S

### Removing & reinstalling bottom cover.

With the unit still hooked up, start by disconnecting the cap from the chemical detergent pouch and then running a shipping cycle from the set-up menu. This will drain most of the water and detergent from the unit.


If you cannot run a shipping cycle, because the unit is without power, pull the drainage tube out from under the middle of the kickplate and allow it to drain into a waste bottle. (Without power, you may have to manually unlock the door using the door latch.)

Remove the course filter and sump filter and use an absorbent cloth to sop up residual water from the sump. After draining the unit, turn it off, disconnect the power and disconnect all water connections.

Slide an absorbent cloth under the unit to catch any remaining water.


Using the two handle holds located at the front under the kickplate, pull the unit towards you and tip it onto its back. On the bottom cover there are 2 small rubber standoffs connecting the dryer inlet duct to the bottom cover. Push these through their holes to detach them from the cover. Remove 6 screws and slide the cover up to release the tabs from the chassis.

To reinstall, slide the cover back into position, replace the screws and pull the rubber standoffs back through the cover. (NOTE: if the rubber standoffs are not pulled through the cover, the dryer inlet duct will not function correctly.)

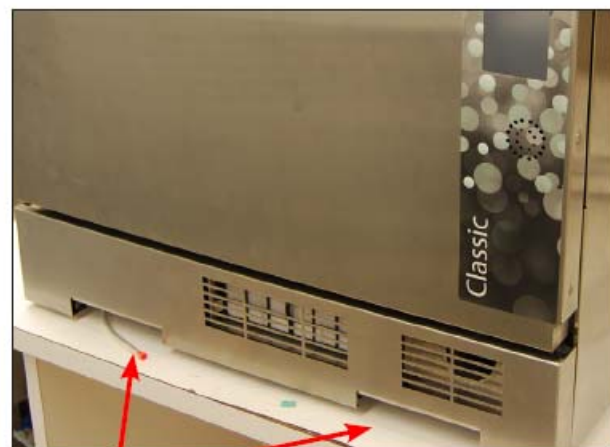


handleholds

Figure 13a

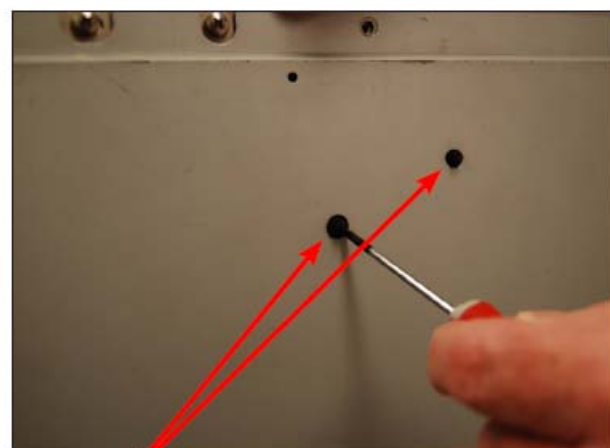

rubber standoffs

Figure 13b

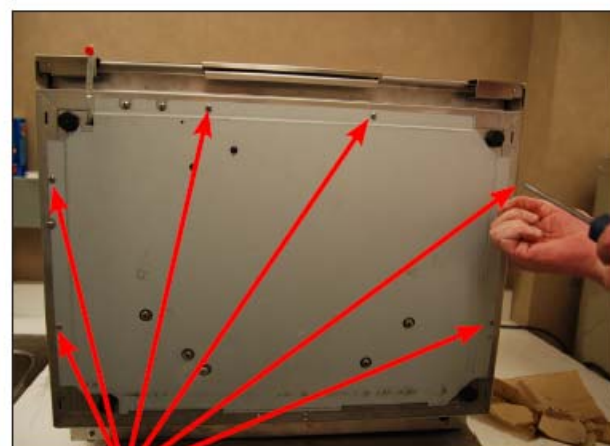

screws

Figure 13c