

# Hydrim C61 G4

## Troubleshooting CF 15

### Chamber Overflow

The water reservoir overflow switch is triggered and did not turn off in 5 seconds (software 406 and lower) or 10 seconds (software 407 and higher).

Check software if 406 or lower update to latest software.


Check that hot and cold inlet valves are turning off.

Check the full & overflow switches. If switches are bad replace air gap.




Cold Water Inlet

Hot Water Inlet

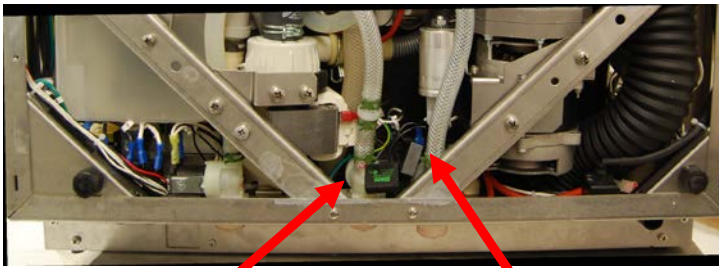

Hot Water Inlet  
01-113330S

Cold Water Inlet  
01-113331S

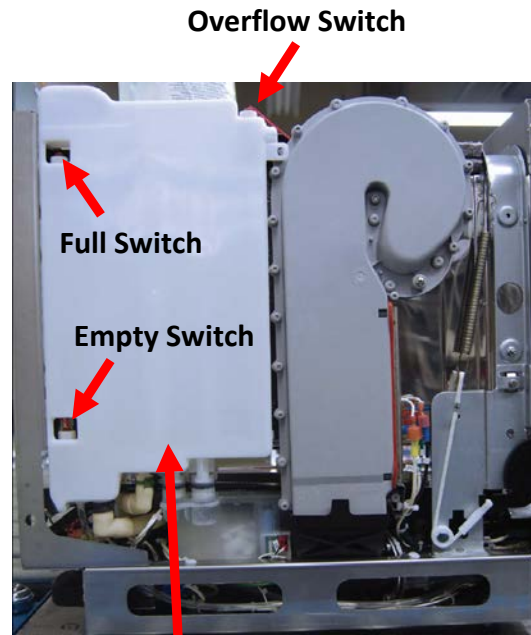

Overflow Switch

Full Switch

Empty Switch

Air Gap  
(Backflow Preventer)  
01-113318S