


## Hydrim C61W G4 CF23 & CF 24 Top & Lower Wash Arm RPM Error

Remove the wash arms by pulling down on the top arm and up on the lower arm. Check all of the spray ports in the arms to make sure they are not clogged. Shake the arms and listen for foreign objects rattling in the arms. Clean or replace the arms as needed. (Wash Arm part #01-113302S)

Check the strainer assembly for debris. Turn the thumb screw counterclockwise and remove the strainer assembly. Clean as needed. (Strainer Assembly part #01-113332S)

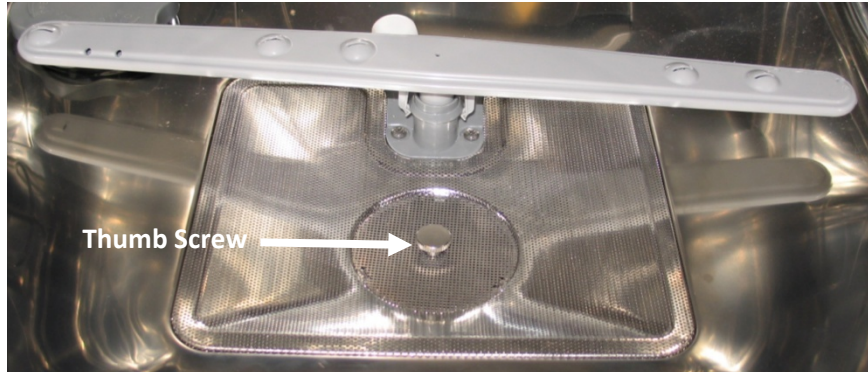

Check the fixed screen, sump for debris. Remove the Phillips screw from the center of the screen and the two Phillips screws holding the lower wash arm mount. Remove the fixed screen, sump and clean as needed. (Fixed Screen, Sump part #01-113387S)

