

Hydrim C61 G4 Chemical Bracket Removing & Reinstalling

Removing & Reinstalling the Chemical Bracket

Remove top cover.

Remove right side cover.

Remove the chemical detergent drawer and bag.

Remove 1 screw at the back of the reservoir filling pump bracket.

Remove 6 screws on the chamber fascia.

Loosen the 3 screws at the base of the chemical bracket.

Slide it towards the back to release it from the screws at the base and pull it up and out from the unit.

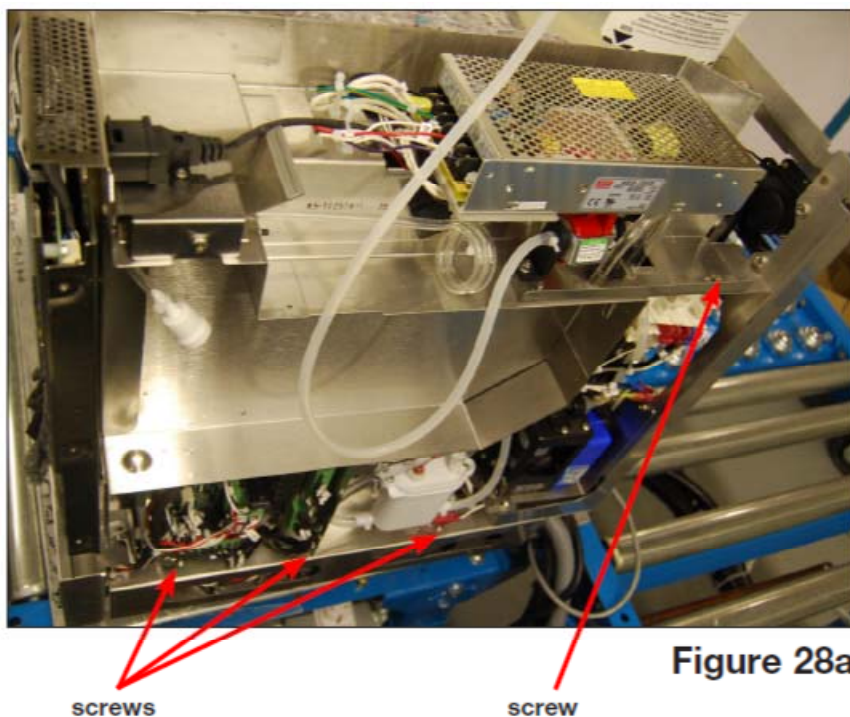


Figure 28a

To reinstall:

Slide the bracket back into position, engaging the 3 screws at the base.

Fasten the 6 screws on the chamber fascia.

Fasten the 1 screw at the back of the reservoir filling pump bracket.

Reinstall the chemical detergent drawer and bag, the right side cover and the top cover.

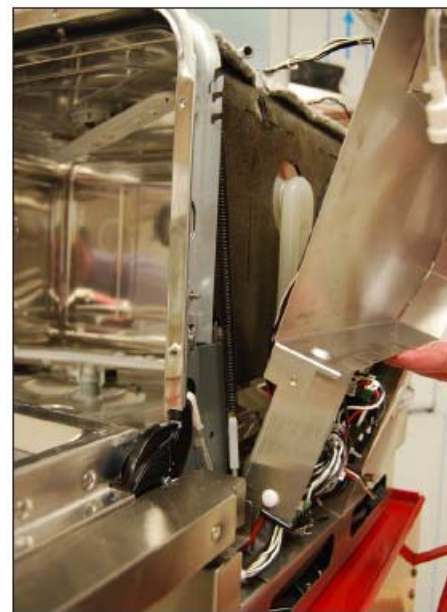


Figure 28b

