
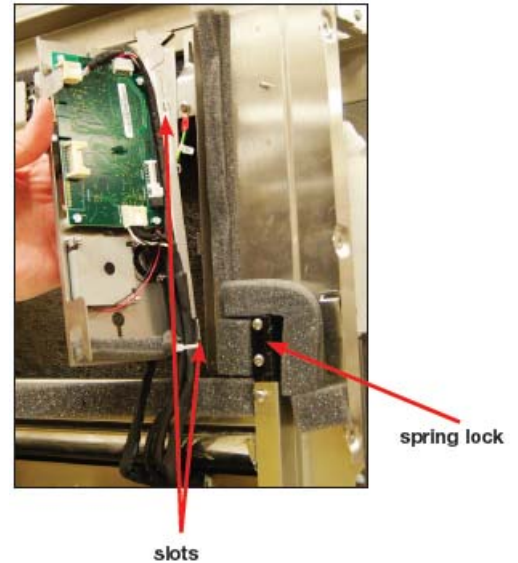


## Hydrim C61 G4 Door Fascia Removing & Reinstalling Part #01-113294S

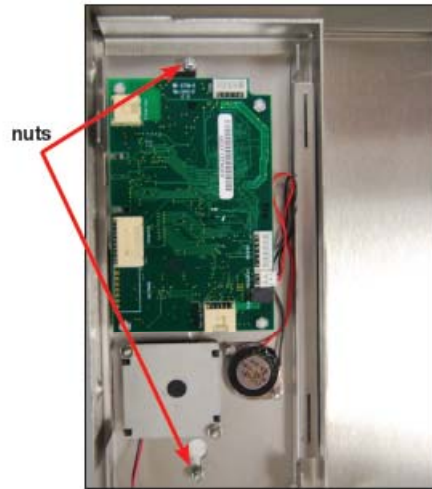

### Removing & reinstalling the door fascia.

Open the door and remove 3 screws on right side of door, 2 on inside left and 2 on the inside top. Slide the door cover up just enough to disengage it.

**Caution:** the LCD and controller are attached to the door cover.

Pull it up and out from the top and rest it on the work surface in front of the unit to access the LCD bracket.

Unscrew the two nuts fastening the LCD bracket to the door cover and lift the bracket out.

Hook the LCD bracket into its service position on the door (it hooks into two slots and one spring lock at right on door).

To reinstall, remove the LCD bracket from its service position on the door, place it into position in the door cover and fasten the two retaining nuts.

Bring the cover into position onto the front of the door and slide it down into place.

Replace the 3 screws on the right side of the door, the 2 on the inside left and the 2 on the inside top.