

Hydrim C61 G4 Door Latch Removing & Reinstalling Part #01-113322S

Removing & reinstalling the door latch assembly.

Remove the door fascia.

Remove the 3 screws on the latch assembly (2 screws at the inside top of the door and 1 at the right of assembly). (Figure 20a)

Disconnect the wires on the latching microswitch (wires 69 & 70), the filter (wires 67 & 68), and the door latching solenoid (wires 93 & 94). (Figure 20a & 20b)

Lift the double-sided tape to remove capacitor, if required.

To reinstall:

Reattach all the wires.

Re-stick capacitor to door latch assembly.

Fasten the 2 screws at the top of the door latch.

CAUTION: door has sharp edges.

Fasten the 1 screw into bracket at right.

Fasten cable ties.

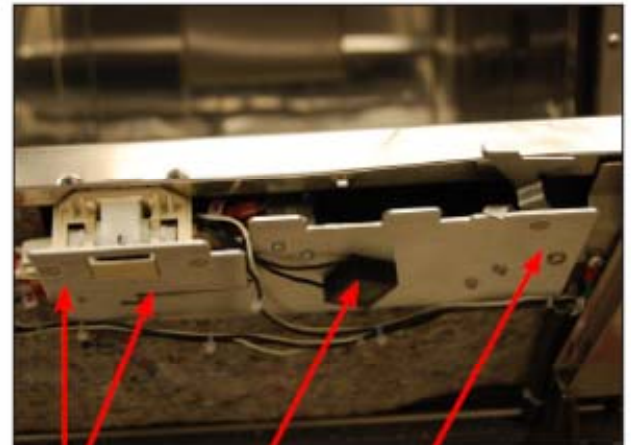


Figure 20a
screws filter screw

door latching microswitch

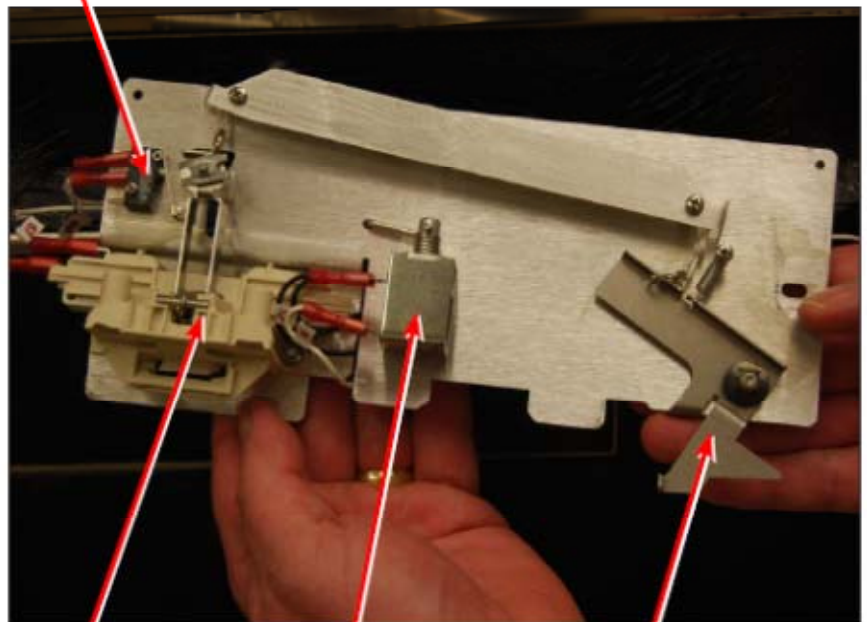


Figure 20b
door latch door latching solenoid manual door latch