

# Hydrim C61 G4 Door Microswitch & Solenoid Removing & Reinstalling Door Latch Microswitch Part #01-113605S Door Latch Solenoid Part #01-113319S

door latching microswitch

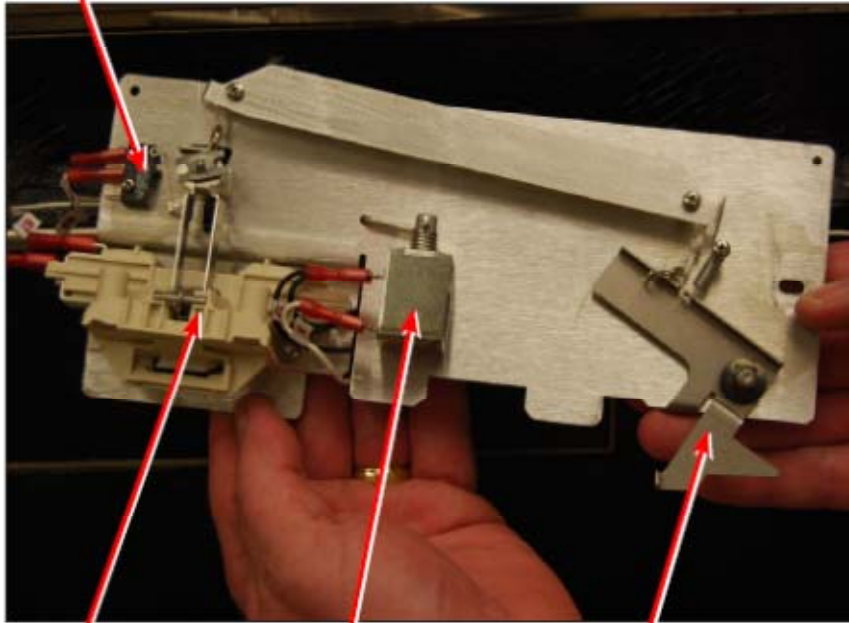


Figure 20b

door latch

door latching solenoid

manual door latch

## Removing & reinstalling the door latch microswitch

Remove the door latch assembly.

Unscrew the 2 screws fastening the door latch microswitch to the door latch assembly and remove.

To reinstall:

Replace the door latch microswitch and fasten with the 2 screws.

Reinstall the door latch assembly and the door fascia.