

Hydrim C61 G4 Door Springs Removing & Reinstalling Part #01-113298S

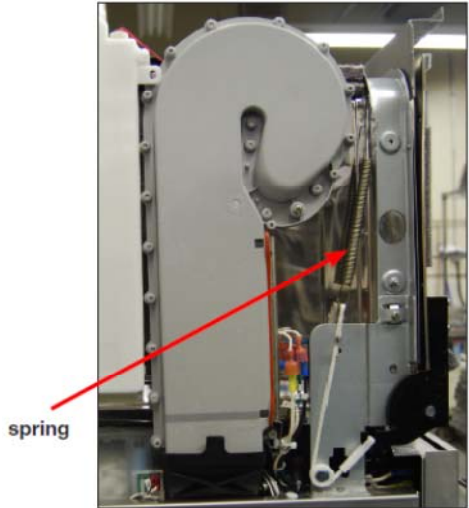


Figure 17a

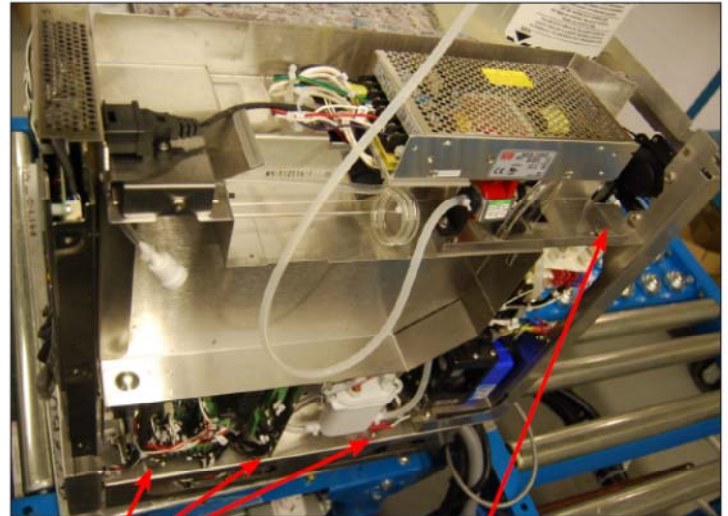


Figure 17b



Figure 17c

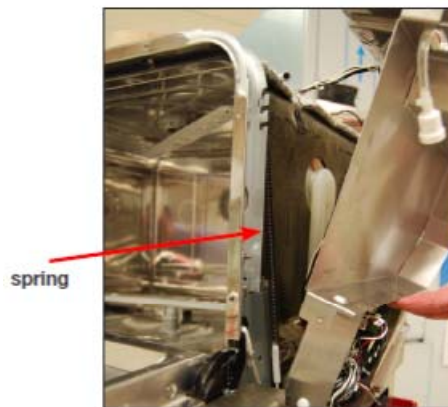


Figure 17d

Removing & reinstalling the door springs. (kit has set of 2 springs & rope)

To remove the left side door spring:

Remove top cover.

Remove left side cover.

Unhook the door spring to replace (Figure 17a)

To reinstall, reverse procedure.

To remove the right side door spring:

Remove top cover.

Remove left & right side covers (Left side cover must be removed to remove chamber fascia).

Remove 1 screw at rear of reservoir refill pump bracket. (Figure 17b)

Remove 6 screws on chamber fascia and remove chamber fascia. (Figure 17c)

Loosen 3 screws at base of chemical bracket. (Figure 17b)

Slide chemical bracket to rear and tip it back to access right door spring.

Unhook the door spring to replace. (Figure 17d)

To reinstall, reverse procedure.