

Hydrim C61 G4 Dryer Assembly Removing & Reinstalling Part #01-113324S

Removing & Reinstalling the Dryer Assembly

Remove the top and left side cover.

From inside the chamber, remove 4 screws holding the dryer outlet cover.

Remove the screw connecting the dryer assembly to the air gap.

Disconnect the dryer heater wiring.

Pull the dryer free from the rubber air intake boot.



Figure 33a

dryer outlet
cover



screw

dryer boot

Figure 33b

To reinstall:

Put the dryer assembly in place.

Reattach the rubber air intake boot.

Attach the dryer outlet cover on the chamber side and fasten with 4 screws.

Reattach the screw connecting the dryer assembly to the air gap.

Reattach the dryer heater wiring.

Reinstall covers.