

# Hydrim C61 G4 Dryer Heater

## Removing & Reinstalling

### Part #01-113273S

#### Removing & Reinstalling the Dryer Heater

Remove the top & left side cover.

Disconnect the heater wiring.

Unscrew the 2 screws.

Cut the wiring cable tie.

Remove the door spring and tuck the wiring harness out of the way under the chamber to improve access.

Slide the heater out of the dryer duct housing.

To reinstall:

Put the heater back in the dryer assembly.

Insert the screws and do not over tighten.

Reconnect the wiring.

Reinstall covers.

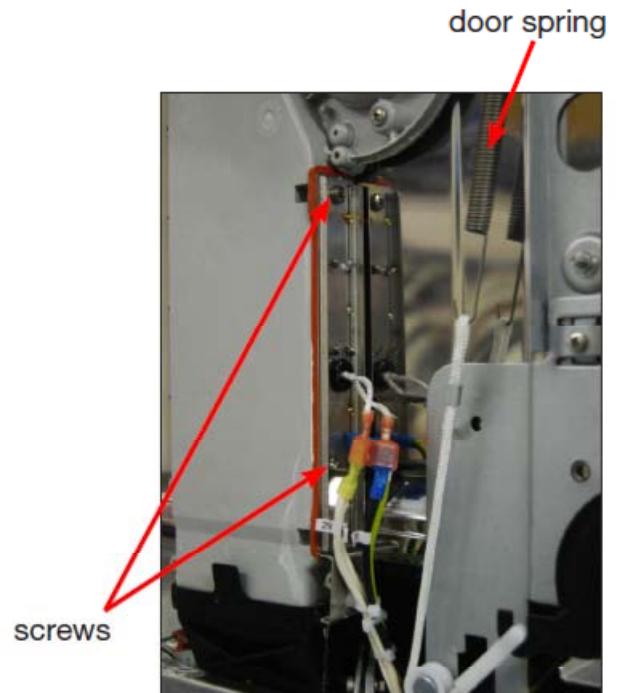


Figure 32