

Hydrim C61 G4 Dryer Motor Removing & Reinstalling Part #01-113285S

Removing & Reinstalling Dryer Motor

Disconnect the detergent bag and remove bag and drawer.

Run a shipping cycle to drain the unit, disconnect power and flip unit onto its back.

Remove bottom cover.

Remove diagonal brace.

Remove 2 screws at the chassis and 3 screws on the dryer motor bracket. (Figure 45a)

Disconnect dryer exhaust from exhaust outlet.

Pull dryer motor out with dryer boot attached. (Figure 45b)

Disconnect wiring.

Detach dryer boot from dryer motor.

To reinstall:

Connect wires.

Put dryer motor into place.

Thread dryer exhaust back into position.

Fasten top of bracket to chassis with 3 screws.

Reattach the dryer motor exhaust boot to dryer.

Reinstall bottom cover. To reinstall, slide the cover back into position, replace the screws and pull the rubber standoffs back through the cover. (**NOTE:** if the rubber standoffs are not pulled through the cover, the dryer inlet duct will not function correctly.)

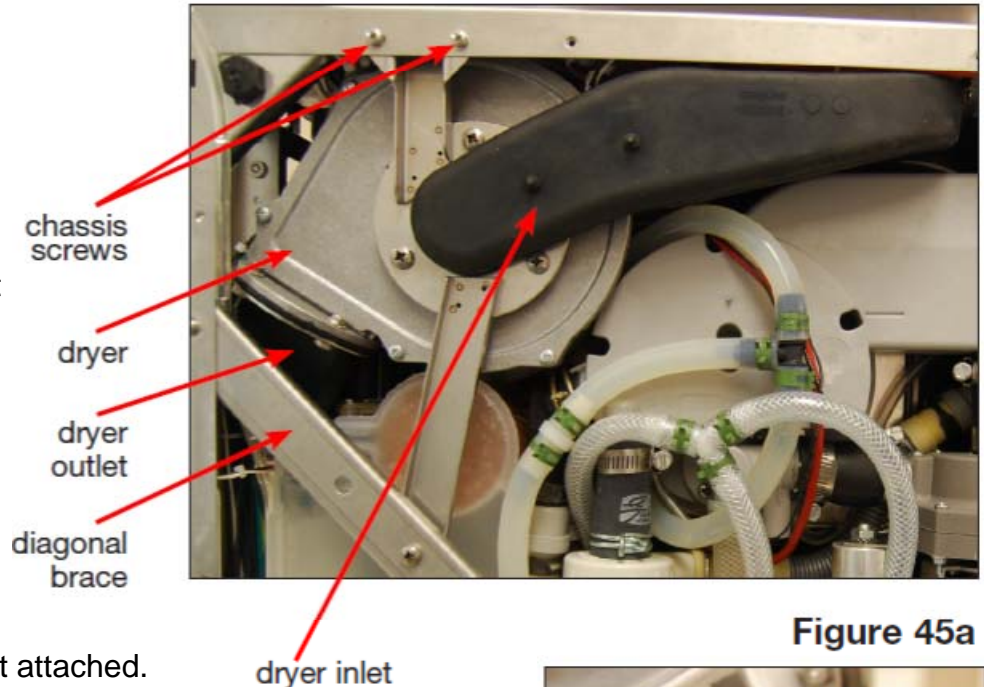


Figure 45a



Figure 45b

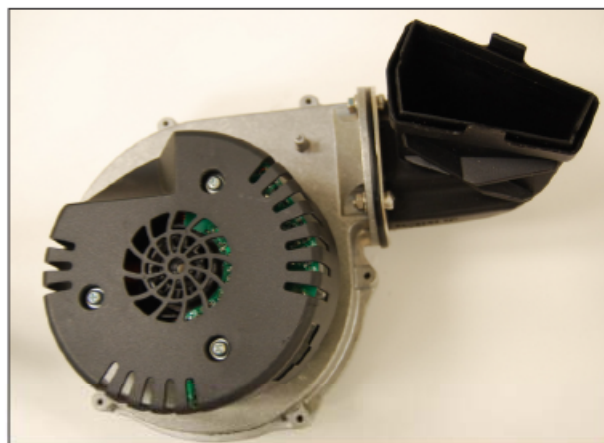


Figure 45c