

Hydrim C61 G4 Exhaust Duct Removing & Reinstalling Part #01-113335S

Removing & Reinstalling the Exhaust Duct

Power off unit and disconnect power cord.


Remove top & rear covers.

From inside the chamber, remove the 2 screws to release the splash shield. (Figure 37a)

Using a 7mm nut driver, remove the exhaust duct mounting ring. (Figure 37b)


Push down gently on the exhaust duct and tip it back to access the hose clamp on the exhaust duct hose. (Figure 37c)

Loosen clamp on hose and slide over the hose to remove the exhaust duct.

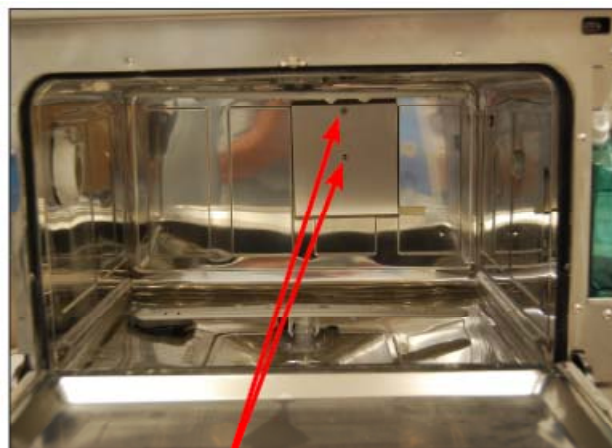


screws

Figure 37a

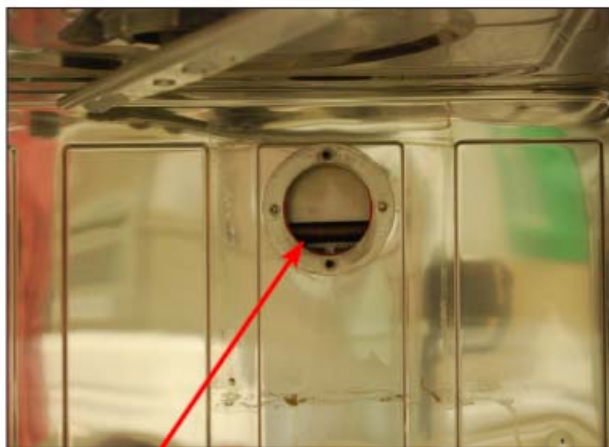

mounting ring

Figure 37b

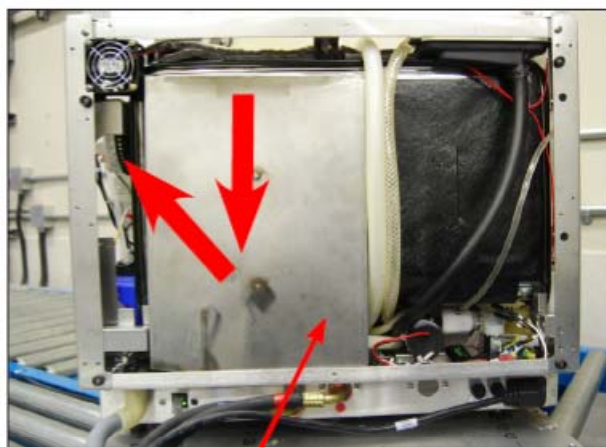

exhaust duct

Figure 37c

To reinstall:

Attach the exhaust duct to the hose, tightening the clamp.

Put the exhaust duct back into place, aligning the screw posts to fit the holes in the chamber.

From inside the chamber, replace the mounting ring and fasten the 2 retaining nuts.

Reinstall the splash shield.

Reinstall the covers.



Figure 37d