

Hydrim C61 G4 Kickplate Removing & Reinstalling Kickplate Part #01-113292S Filter Gasket Parts #01-113262S

Removing & reinstalling the kickplate.

Caution: cover sharp edges on the filter cutout with tape to protect hands.

Open the kickplate and remove HEPA filter.

Detach the dryer inlet duct by pulling off the HEPA filter gasket and pulling the dryer inlet duct out of the groove so that you can push it free of the kickplate.

Unscrew the 2 magnets by hand and remove the 2 screws.

Unclip the plastic edge guard at the bottom right of the door and pull the wire harness free of the kickplate.

Remove the metal panel between the power switch and the HEPA filter to detach the microswitch wires at right. With the condenser exhaust still attached, support the kickplate door to keep it from falling and pull the kickplate off.

Detach the hose clamp on the exhaust duct hose and pull the kickplate free.

To reinstall, Attach the exhaust duct hose using the hose clamp.

Attach the HEPA filter microswitch wires and push the kickplate into place.

Reattach the 2 magnets and the 2 bumpers with screws.

To reattach the dryer inlet duct, pull it through the kickplate and push it back to hook its edges into the grooves at the top and bottom of the dryer inlet duct cutout and over the tabs at the left and right of the cutout.

NOTE: Be certain the gasket is properly positioned in the dryer inlet duct or replace with a new gasket.

