

# Hydrim C61 G4 LCD Fan & Speaker Removing & Reinstalling LCD Fan Part #01-113284S Speaker C61 Part #01-113682S

## Removing & reinstalling LCD Fan.

Remove the LCD bracket.

Remove 2 screws affixing the fan to the bracket.

Remove the fan.

To reinstall, reverse procedure.

## Removing & reinstalling the speaker.

Remove the LCD bracket.

Remove the 2 screws affixing the speaker to the bracket.

Remove the speaker.

To reinstall, reverse procedure.

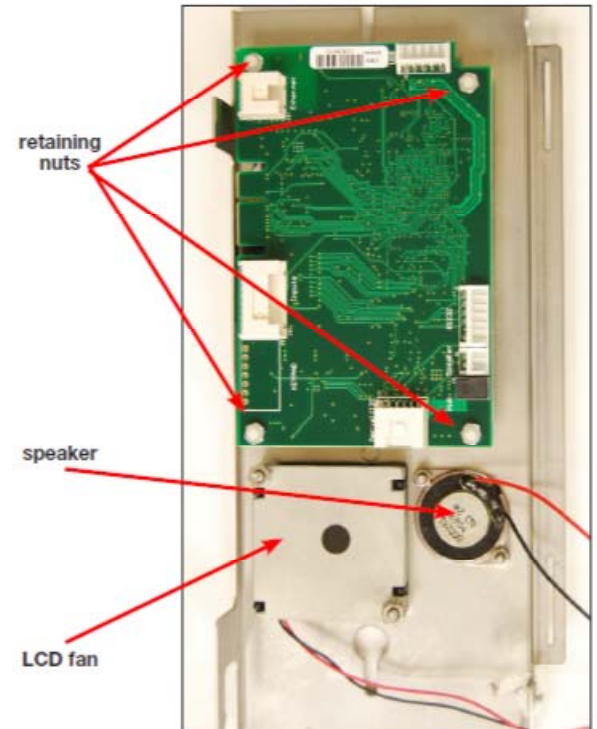


Figure 19a

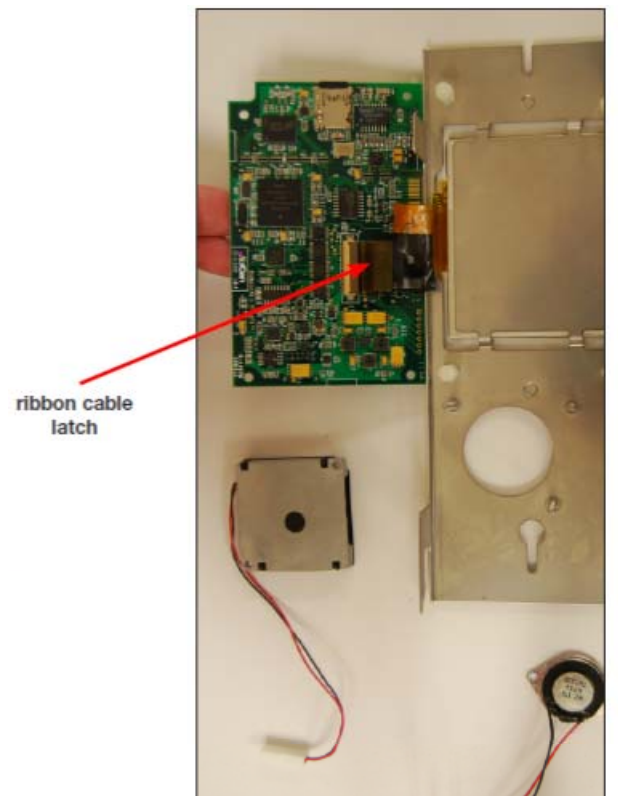


Figure 19b