


Hydrim C61 G4 Left Side Cover Removing & Reinstalling Part #01-113288S

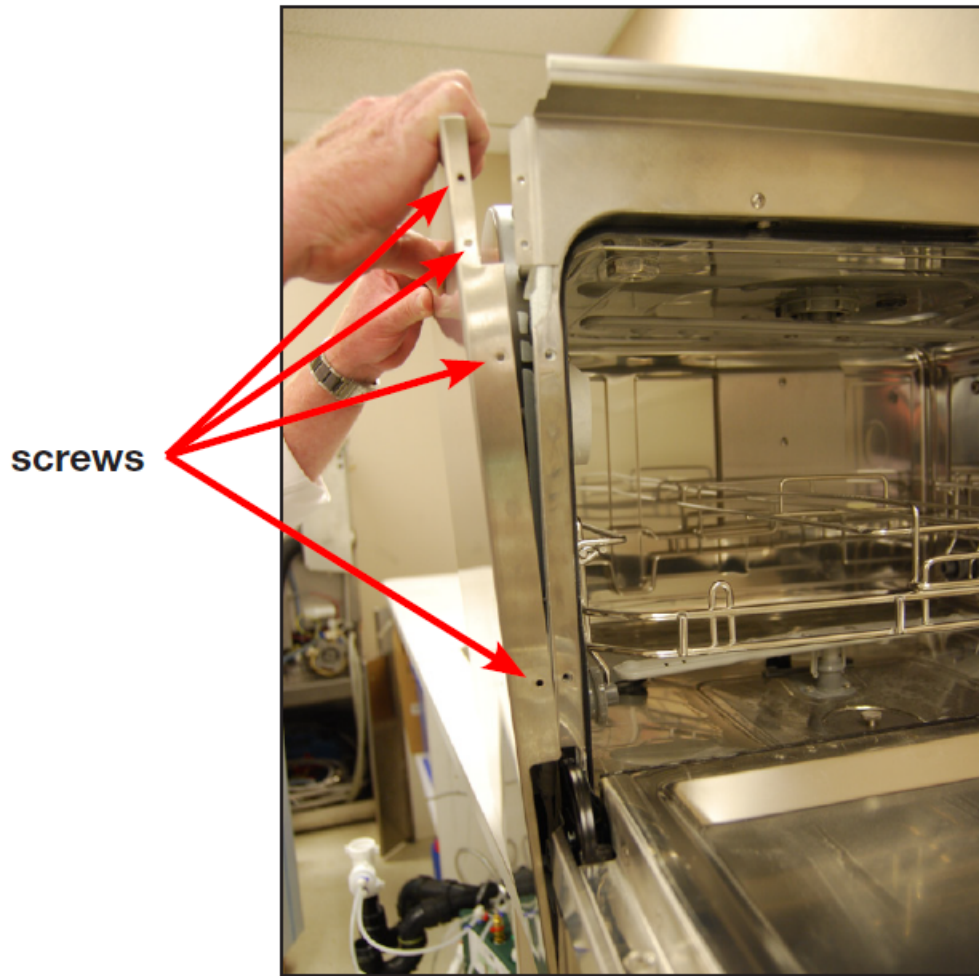

Removing & reinstalling the left side cover.

Remove the top cover

Open the chamber door to remove 4 screws in the front.

Remove 3 screws on the back of the unit.

Tip the cover back and slide it down to disengage two tabs that insert into the chassis at bottom.

To reinstall, bring the cover into position slightly lower than the unit to be able to engage the two bottom tabs and the one at the top. Slide the cover up into place and replace 4 screws.

Replace the top cover.