


Hydrim C61 G4 Power Switch Removing & Reinstalling Part #01-112024S

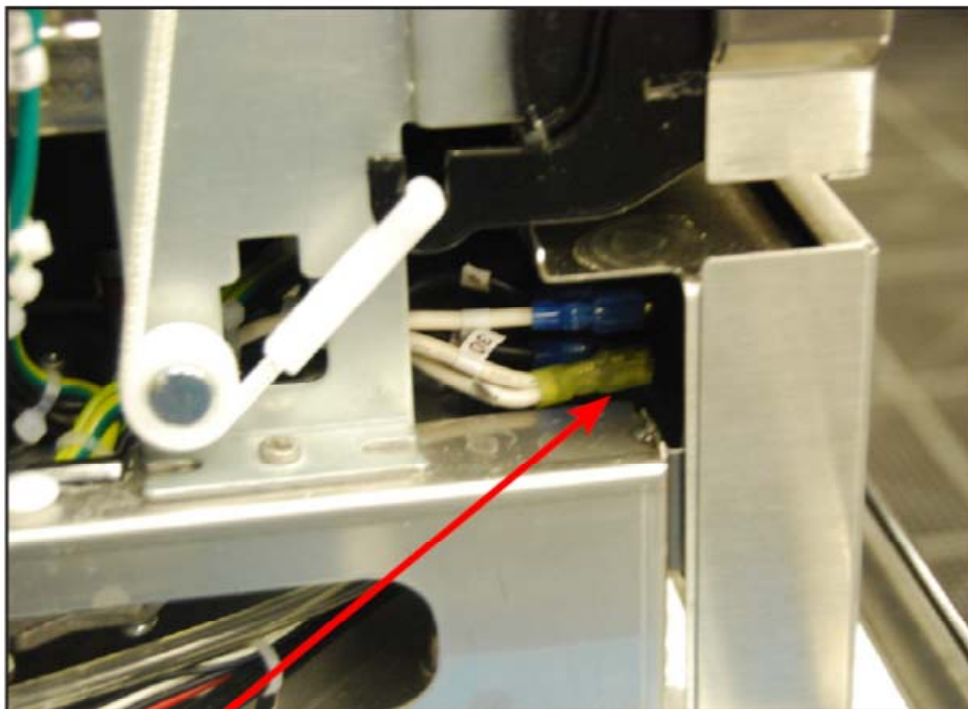

power switch contacts

Removing & reinstalling the power switch

Remove top cover and left side cover.

Remove two contacts from the back of the power switch using needle nose pliers. This will help to pull the power switch through the cutout in the chassis.

Carefully pry the switch out using a flat head screwdriver.

Remove all the remaining contacts.

NOTE: Switch wire connectors can also be accessed by removing steel plate in front of kickplate.

To reinstall, reattach contacts to power switch according to table below and push into cutout in chassis.

Power switch Terminal position	Corresponding wire
1	3 BLU
1A	29/30 YEL
2	4 BLU
2A	5 BLU

Reinstall left side cover and top cover.