


## Hydrim C61 G4 Right Side Cover Removing & Reinstalling Part #01-113290S

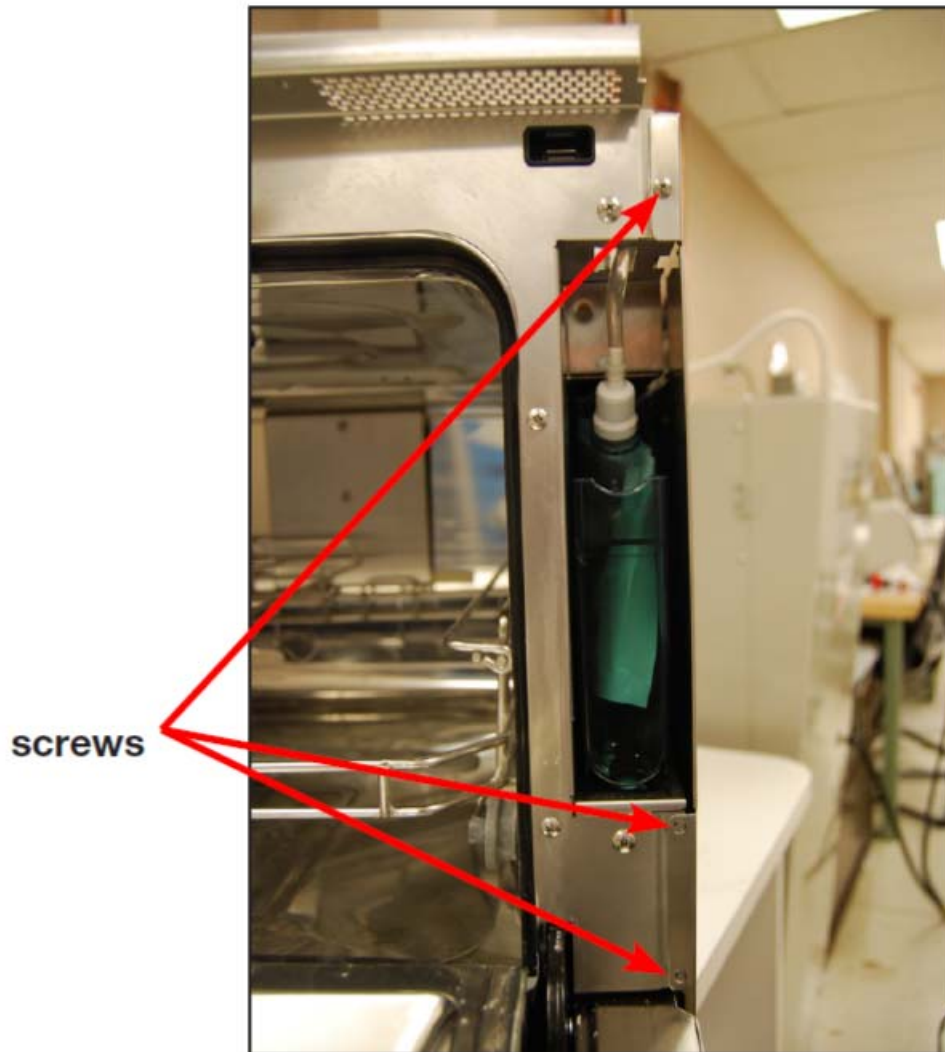

### **Removing & reinstalling the right side cover.**

Open the chamber door to remove 3 screws in the front.

Remove 3 screws at the back of the unit. (Not shown)

There are 2 tabs at the bottom of the cover. To disengage these from the chassis, pull the cover out from the top and slide it down.

To reinstall, bring the cover into position slightly lower than the unit to be able to engage the two bottom tabs. Slide it into position and push it into place at the top. Replace 3 screws at the back and 3 screws at the front.