

# Hydrim C61 G4 Sump Temperature Sensor Removing & Reinstalling Part #01-113271S

## Removing & Reinstalling Sump Temperature Sensor

Disconnect the detergent bag and remove bag and drawer.

Run a shipping cycle to drain the unit, disconnect power and flip unit onto its back.

Remove bottom cover.

Remove the drain pump.

Inside chamber, remove coarse filter and fine filter to access sump.

Unscrew the nut on the inside by hand and pull it out from the unit bottom.

Disconnect wires 71 & 72 from the wiring harness.

sensor  
wire

Note: drain  
pump removed

sensor with nut



Figure 43b

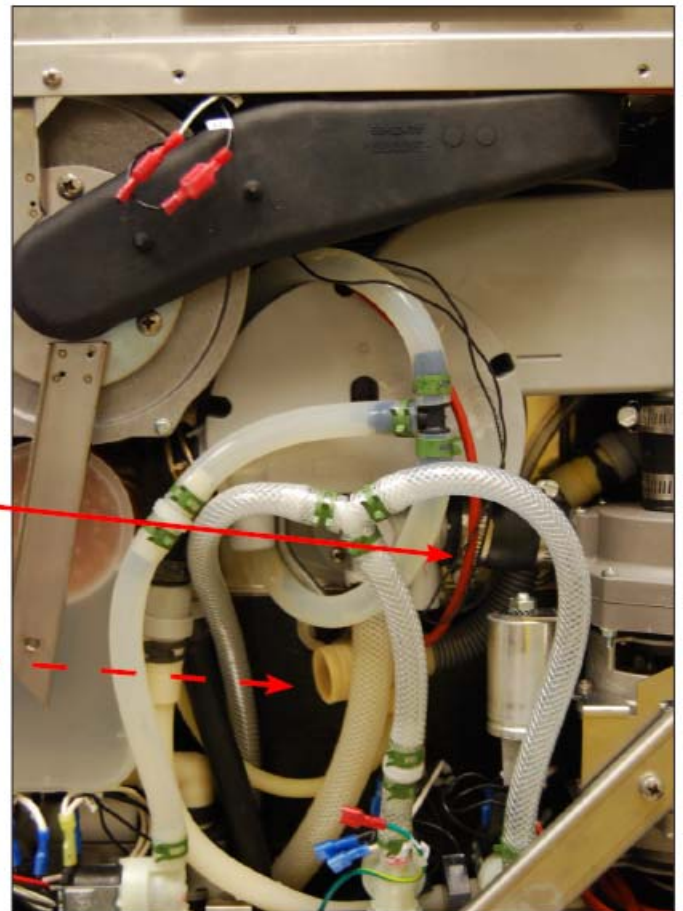


Figure 43a

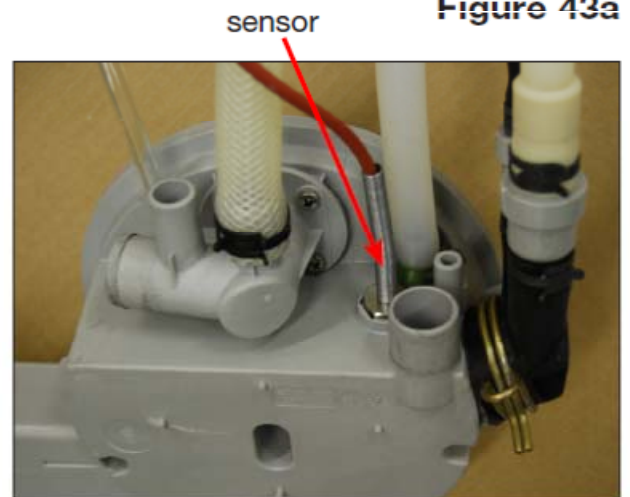


Figure 43c

To reinstall:

Reconnect the wires.

Hand-tighten the sensor into position.

Reinstall the drain pump.

Reinstall bottom cover. To reinstall, slide the cover back into position, replace the screw and pull the rubber standoffs back through the cover. (**NOTE:** if the rubber standoffs are not pulled through the cover, the dryer inlet duct will not function correctly.)