

Hydrim C61 G4 Upper Wash Arm Inlet Removing & Reinstalling Part #01-113681S

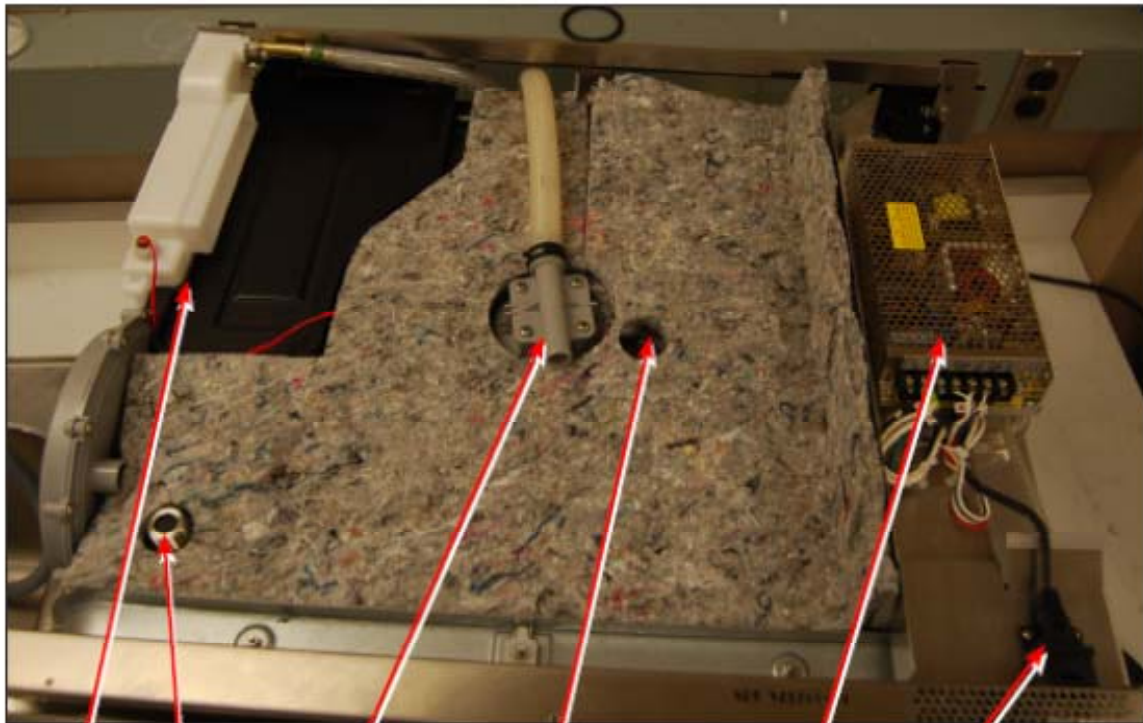


Figure 38

air gap overflow tray test port upper wash arm inlet upper wash arm sensor DC power source USB port

Removing & Reinstalling the Upper Wash Arm Inlet

Remove the top cover.

Pull down the upper wash arm inside the chamber to remove it.

Unscrew the mounting nut to release the upper wash arm inlet.

To reinstall:

Reconnect the upper wash arm inlet.

Put the upper wash arm inlet into position.

Fasten the mounting nut on the inside.

Reinstall the upper wash arm by pushing it into position.

Reinstall the top cover.