

Hydrim C61 G4 Upper Wash Arm Sensor Removing & Reinstalling Part #01-113681S

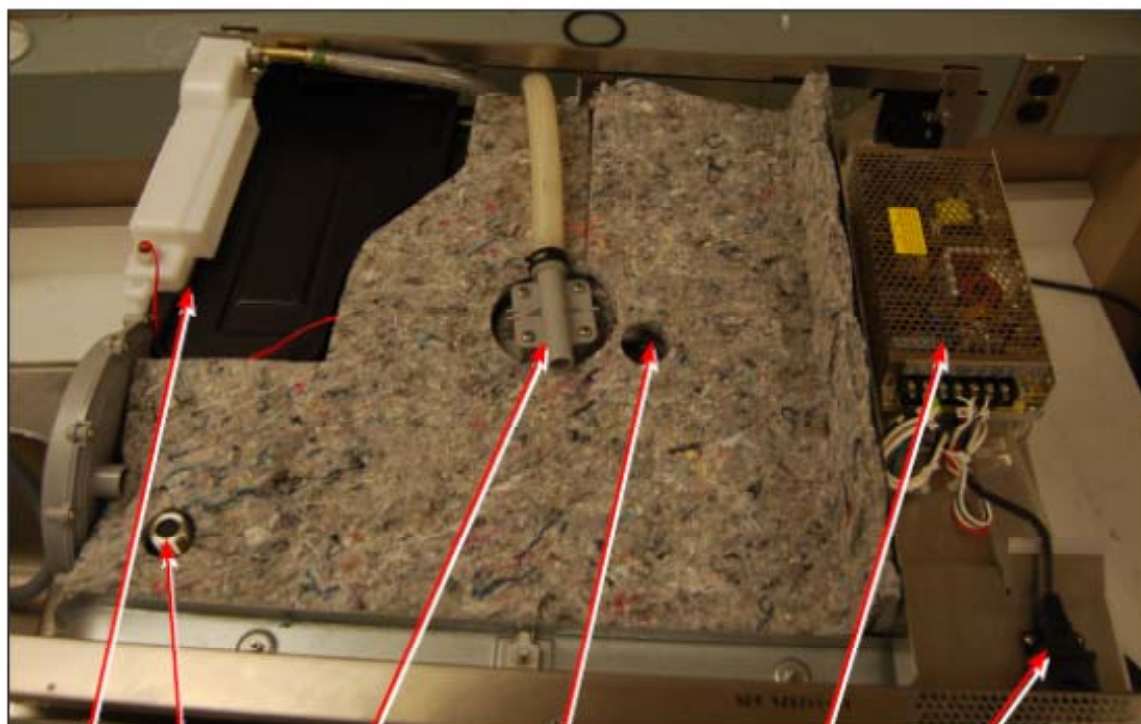


Figure 38

air gap overflow tray test port upper wash arm inlet upper wash arm sensor DC power source USB port

Removing & Reinstalling the Upper Wash Arm Sensor

Remove the top cover.

Disconnect black wires 73 & 74.

Unscrew the mounting nut to release the sensor.

To reinstall:

Place the sensor into position and hand tighten the mounting nut.

Reconnect the black wires 73 & 74 to the wiring harness.

Reinstall top cover.

