

Hydrim C61 G4 Water Heater

Removing & Reinstalling

Part #01-113272S

Removing & reinstalling Water Heater

Disconnect the detergent bag and remove bag and drawer.

Run a shipping cycle to drain the unit, disconnect power and flip unit onto its back.

Remove bottom cover.

Disconnect the dryer exhaust tubing from the dryer outlet on the kickplate and pull it out of the way to access the heater.

Remove water heater wires.

Remove grounding wire nut.

Remove 6 screws fixing heater to sump.

Pull the heater out gently.

To reinstall:

Insert gasket into gasket recess first, then insert heater and replace screws.

Connect wires. (Figure 41a)

Reattach dryer exhaust tubing to the outlet in kickplate.

Reinstall bottom cover. To reinstall, slide the cover back into position, replace the screws and pull the rubber standoffs back through the cover. (**NOTE:** if the rubber standoffs are not pulled through the cover, the dryer inlet duct will not function correctly.)

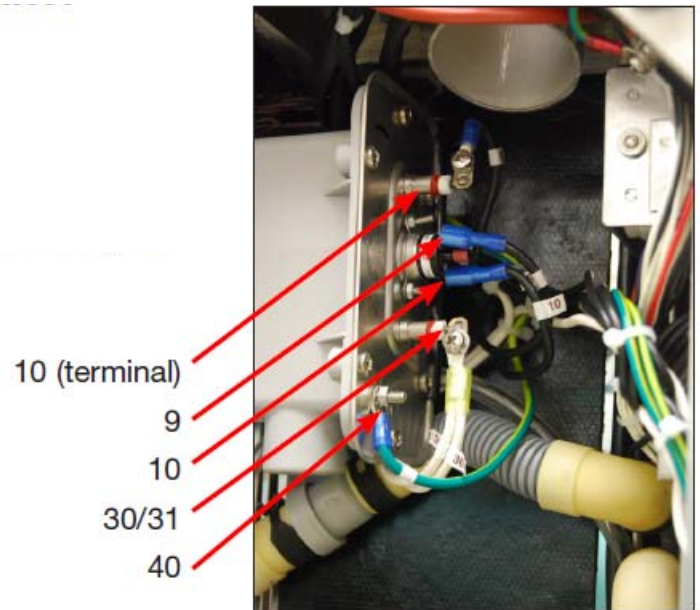


Figure 41a

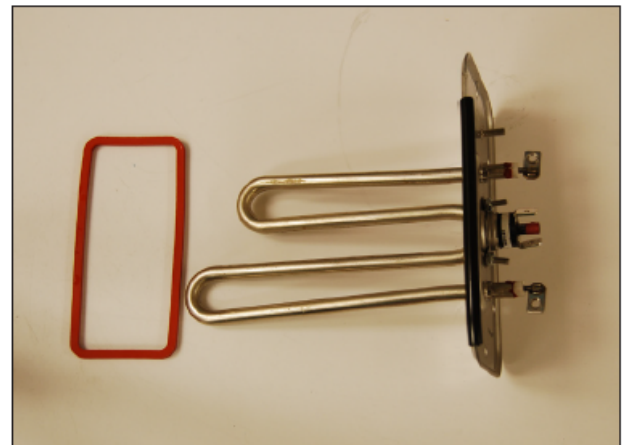


Figure 41b