

Hydrim C61 G4 Water Valves Removing & Reinstalling Part #01-113330S

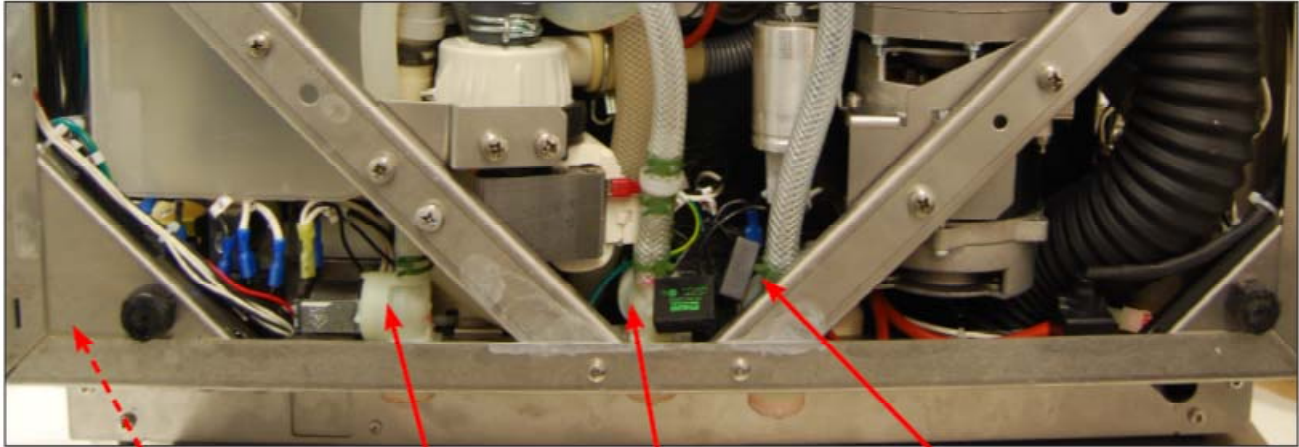


Figure 47

A/C power contacts
(not shown)

RO valve

hot water valve

cold water valve

Removing & Reinstalling the Water Valves

Valves for hot water, cold water and R/O water (if fitted) inlets can be accessed from the unit bottom. Run a shipping cycle to drain the unit, disconnect the power, and flip the unit on its back.

Remove bottom cover.

Valves are accessible from this position. (Figure 47)

Unscrew valve from chassis to remove.

Reinstall, reverse instructions.

To reinstall the bottom cover, slide the cover back into position, replace the screws and pull the rubber standoffs back through the cover. (**NOTE:** if the rubber standoffs are not pulled through the cover, the dryer inlet duct will not function correctly.)

Valve	Color Coding
Cold water inlet	Blue
Hot water inlet	Red
R/O water inlet (if fitted)	Yellow

