

Hydrim C61W G4

Replace Air Filter

01-113277S

When the message “Replace air filter” appears the HEPA filter is in need of changing. To change the filter, proceed as follows:

1. Open the Kickplate below the front door of the unit and turn the power off.



2. Remove the old filter by pulling it from the center.
3. Install the new air filter and close the Kickplate.
4. After replacing the filter, go to the “Reset Drying Counter” screen in the Setup menu and reset the drying counter to zero.
5. The filter must be replaced every 750 drying cycles. The unit will continue to run if the filter is not replaced but the drying performance will decrease.

