


To Keep Your Quantim Running Clean and Effectively Use . . .

QUANTIM[®]

Autoclave cleanser

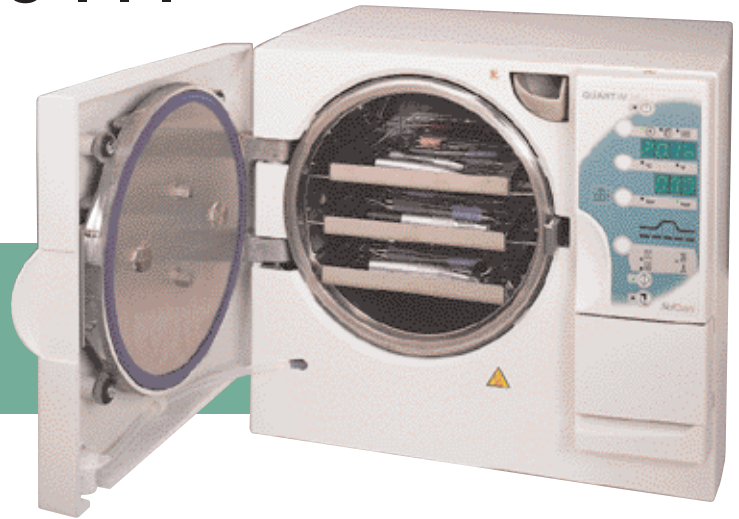

Quantim Autoclave Cleanser is designed to clean the reservoir, chamber and tubing of your QUANTIM Autoclave.

Keeping your Quantim clean is good clinical practice and assists in the function of the unit. Cleaning the inside of the unit is also very important when sterilizing lubricated instruments to eliminate any oily residue.

SciCan recommends that the reservoir, chamber, and tubing be cleaned at least once a week if you are using a recirculating water unit and monthly for single water use units or as required.

After running a cleaning cycle empty the remainder of the water reservoir. Then refill the water reservoir with distilled water and run a cycle to rinse the system.

Directions For Use:

Ensure the reservoir is empty.

1. Add 1/2 of the concentrate to the reservoir. Fill with distilled water and run one complete sterilization cycle. Repeat cycle if necessary
2. Drain the reservoir
3. Refill the reservoir with distilled water and run another cycle to rinse the system. For extremely contaminated systems, or once every three months of normal use, use the complete contents of the concentrate and follow the steps above.

SciCan

SciCan, 1440 Don Mills Road, Toronto, Ontario M3B 3P9
Phone (416) 445 1600 Fax (416) 445-2727

SciCan, Inc., 500 Business Center Drive, Pittsburgh PA, 15205
Phone 412-494-0181 Fax 412-494-4794 Toll free 1-800 572-1211

www.scican.com

QUANTIM is a registered trademark of SciCan, Division of Lux & Zwingenberger Ltd.