

Statim 2000/5000

Rev. 7 PCB

Device Test On/Off

To enter the **Service Setup Mode**, turn power switch ON while holding down Unwrapped and Wrapped buttons.

The **Service Setup Mode** is password protected. A password must be entered to continue. The default password is Unwrapped, Wrapped, Rubber and Plastics, Stop buttons pressed in this order. If the password has been changed the backdoor password is Unwrapped, Wrapped, Unwrapped, Wrapped buttons pressed in this order.

To enter the **Device Test Mode**, press the unwrapped button 11 times.

Device Test On/Off should appear on the top line of the display.

Press the Rubber & Plastics button to accept.

Display should be similar to one shown below:

>Devices Test On/Off Pump Off

Valve Off
Compressor Off
Yellow LED Off
Extra 1L Off
Extra 2L Off
Drawer Relay Off

Keypad:

Unwrapped

Wrapped

Rubber and Plastics

Stop

Select next option. Second line shows the new value

Select previous option. Second line shows the new value

Toggle On/Off selected device

Return to main menu