

Statim 7000 Cover Removal

1. With the unit off, unplug the power cord from the wall outlet and remove the cassette and reservoir from the unit.



2. Remove the water filter or water bypass cartridge (if the unit is using distilled water) from the reservoir area.



3. Remove the eight screws across the bottom front of the unit using a Philips screw driver.



4. Push the cover forward from the back a little to loosen it, until it stops.
5. Detach the LCD/keypad by reaching up inside the cover through the armature opening to feel for a plastic tab located directly behind the Rubber and Plastics key.



6. Push this tab to the left to unlock the LCD/keypad from the cover.




7. Place the LCD/keypad on top of the armature so that it is out of the way as you remove the cover.



8. Release the cover retention clip by reaching inside the cover through the LCD opening and lift up on the tap located on the left most side of the opening. Slide cover forward.



9. Push the cover all the way forward to slide it off from the front.

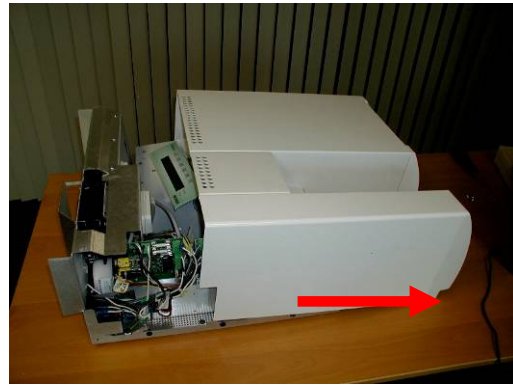

10. To replace the cover. Place the cover back on to the unit about 1 inch away from the back. Once the chassis hooks engage with the cover, push the cover towards the back until the back of the cover is in line with the back of the chassis
11. Replace the LCD panel
12. Insert the 8 screws that were originally removed.