

## Statim G4 Time & Date Setup



### Setting the Time

Touch the   

**Scroll to Time** and select

From the time screen, use the keypad to set the time. Press **EN** to save and  to return to the Setup screen.

To change your unit to display 12-hour time format (24 hour time format is the default setting), go to the Setup menu and use   to scroll to **Time 12/24**, select it and toggle to 12. Press  to save and return to the Setup menu.

## Setting the Date

Touch the  →  → 

**Scroll to Date** and select

From the Date screen, use the keypad to set the date. Press  to save and  to return to the Setup menu. To change your format in which the date appears, return to the Setup menu and use   to scroll to **Date Format**. Select it, and follow the prompts to have the date displayed in the desired format. Press  to save and return to the Setup menu.