

Statim G4

Validation Thermocouple Calibration

Before starting calibration check that there is sufficient distilled water in the reservoir and that the cassette is engaged.

Turn ON the Statim G4 and access the service menu as shown. Select the Information icon, user/technician icon, technician icon and enter the access code 7919 EN. If necessary, scroll thru the options until Calibration appears and select it. Next select V. Calibration and the Statim will run a Validation Thermocouple Calibration Cycle. The display should appear similar to the one below:



The calibration cycle will take approximately 6 minutes to complete. Allow the self-calibration cycle to complete. The temperature within the cassette will rise to sterilization temperature. Wait until the sterilization phase of the cycle ends automatically. The offset value in the upper left-hand corner of the display may have changed to a new offset value. Press the **STOP** button when finished.