

# VistaBrite Troubleshooting

**C51W**

**L110W**


Water Consumption Per Cycle:	C51W	L110W
Rinse and Hold	6 litres / 1.6 gallons	11.5 litres / 3 gallons
Regular Wash	14 litres / 3.7 gallons	19 litres / 4.75 gallons
Heavy Duty Wash	14 litres / 3.7 gallons	25 litres / 6.25 gallons

VistaBrite requires 1.5 to 2 hours to refill the 7 gallon tank (only 4.25 gal of water)

Top gauge is temperature gauge

Middle gauge is inlet water pressure gauge


Bottom gauge is tank water pressure gauge

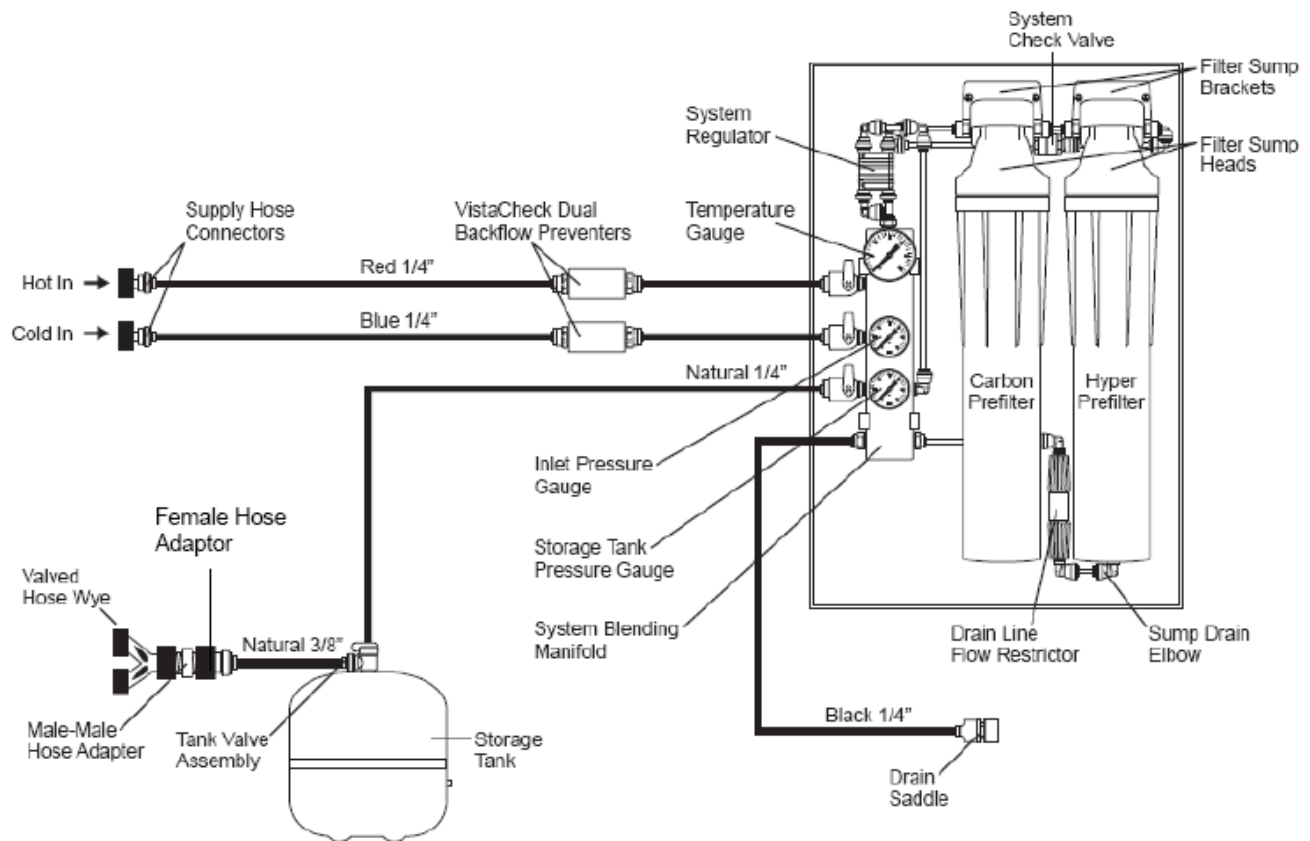

# VistaBrite Type "A" Installation



Utility Water  
Treatment System  
for Instrument Washers

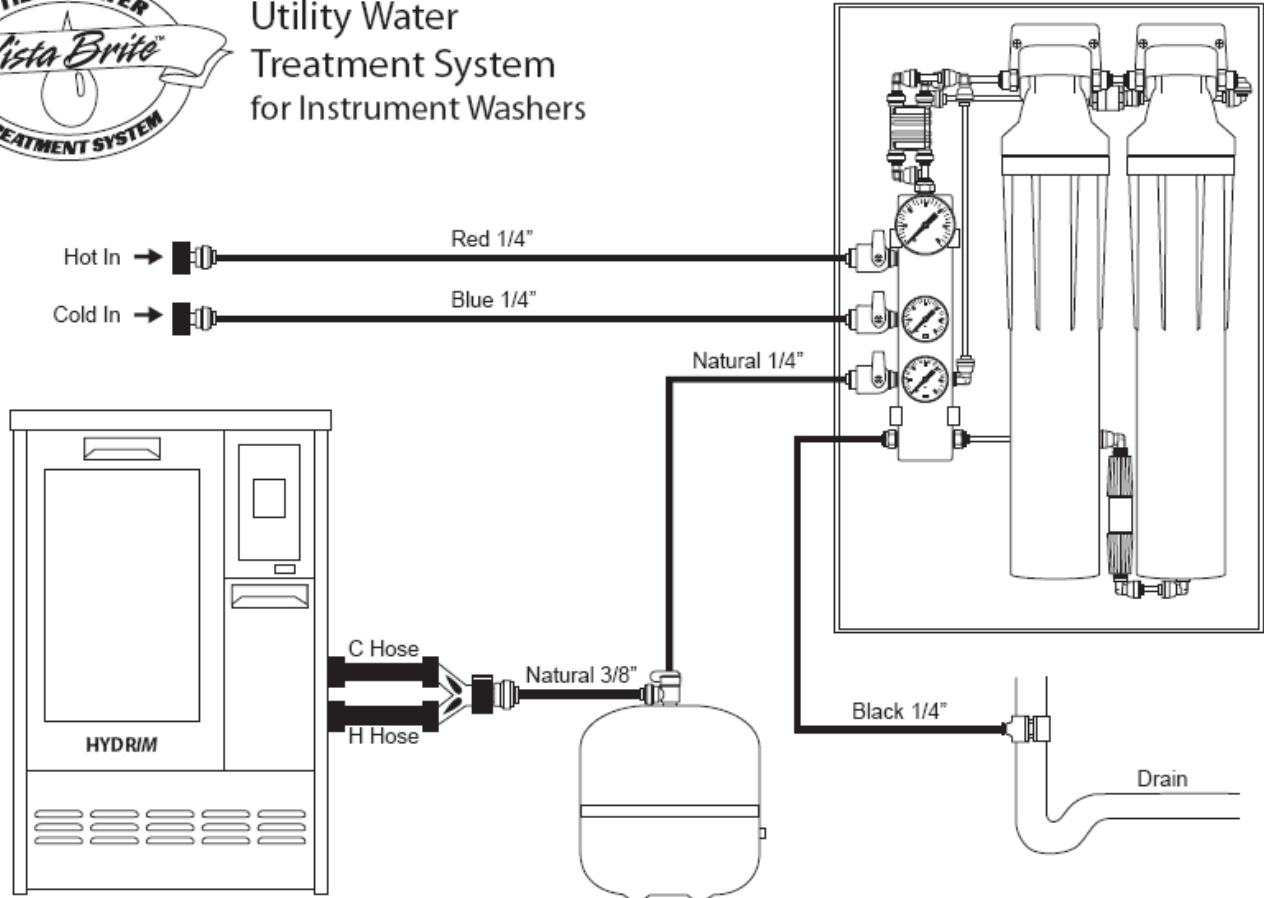


# A

Typical Installation A  
For Optimal Inlet Water Pressure  
(50-80 psi / 3.4-5.5 bar)

# VistaBrite Type "B" Installation



Utility Water  
Treatment System  
for Instrument Washers

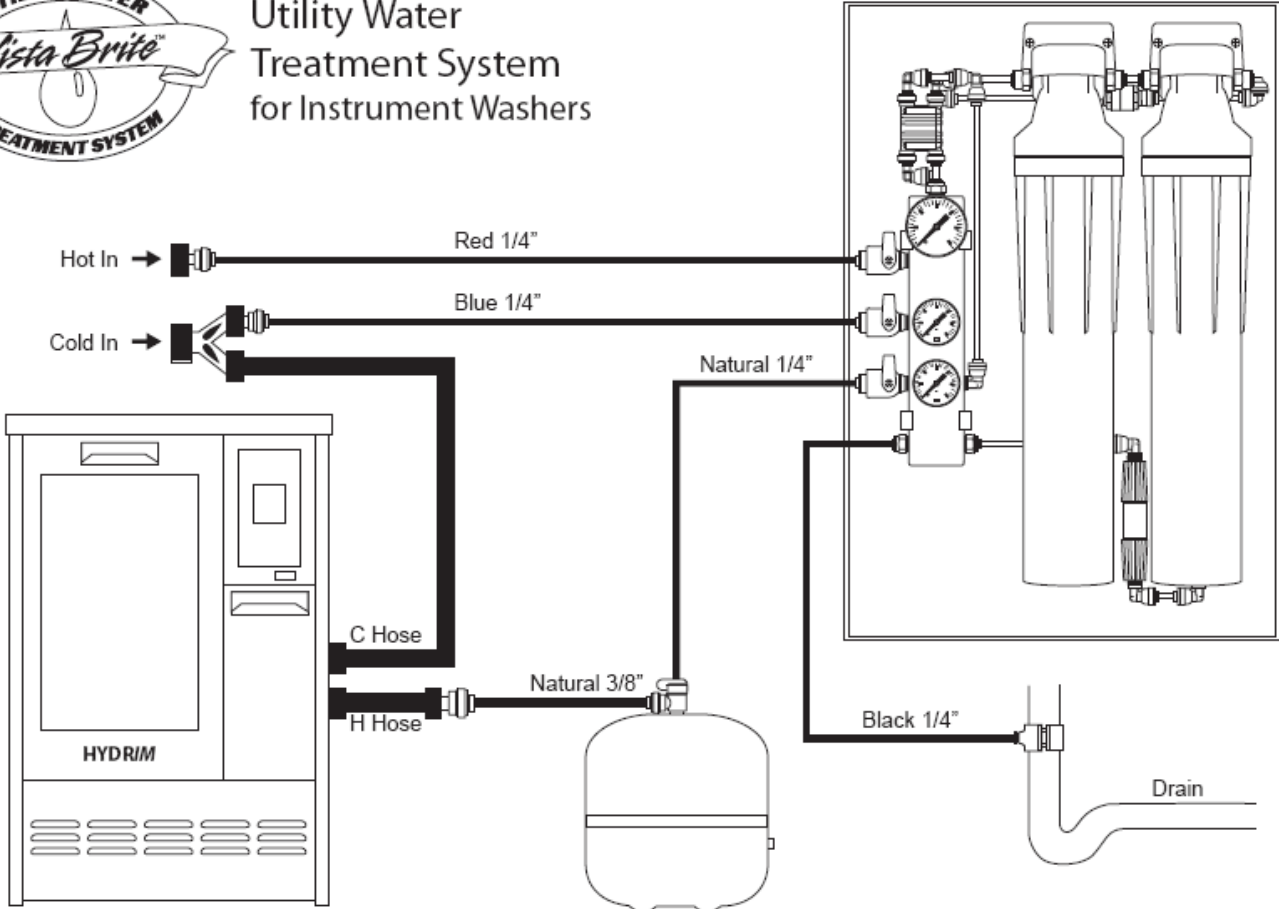

**B**

Typical Installation B  
For Low Inlet Water Pressure  
(40-50 psi / 2.8-3.4 bar)