











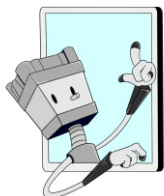


Preventative Maintenance Checklist

Frequency	Preventative Maintenance	Item	Completed by	Date	Order Link
Daily	Clean Door Gasket	    			 Opti1 6"x7" Wipes  Opti-Clave 32oz
	Empty the waste bottle every time you refill the reservoir				
	Check distilled water and waste water bottles				
	Wipe surrounding area with Optim1 disinfecting wipes				
Weekly	Clean the internal part of the tray with dishwashing soap or a mild detergent that does not contain chlorine. After cleaning the tray must be thoroughly washed with water. Dry the tray before reinstalling.				 Enbio S WATER IN Tubing  Enbio S WATER OUT Tubing
	Clean the interior of the process chamber with dishwashing soap or a mild detergent that does not contain chlorine. After cleaning wipe the chamber with a soft cloth until dry.				
	Clean Waste Water Bottle with OptiClave Cleaner				
After 100 Performed Processes	Clean the seal. Use warm water and a microfiber cloth (microfiber with silver particles is acceptable) to clean the device. Use of dull or sharp cleaning tools is not acceptable. Cleaning with chemical agents is not acceptable.				 Enbio S Distilled Water Bottle  Enbio S Waste Water Bottle
	Perform the cleaning when the device has cooled down. After opening the drawer maintain caution and do not bend the drawer. After cleaning, leave the device open until the seal dries.				
Yearly	Periodic maintenance is performed annually, or after 1000 processes (whichever comes sooner) <i>Yearly preventative maintenance needs to be performed by an Enbio Approved Service Technician.</i>				 Enbio S PM Package



Enbio S

StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3



FREE TECH SUPPORT