











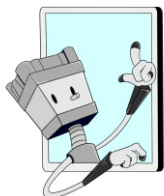


Preventative Maintenance Checklist

Frequency	Preventative Maintenance	Item	Completed by	Date	Order Link
Daily	Clean the door gasket with a wet cloth and check for wear - Replace if necessary.				 Optim1 6"x7" Wipes  Bravo 21V Chamber Drain Filter
	Clean all external surfaces with Optim1 disinfectant.				
	Completely empty the waste tank every time you refill the water reservoir.				
	If using a data logger, transfer the data on the USB drive to your computer.				
	Clean chamber drain filter.				
Weekly	Clean the chamber and appropriate accessories with a wet cloth. If required, use a neutral detergent (a mild detergent containing no alkalis or bleaches). Carefully rinse with distilled water and remove any residue.				 Stat Dri Products  Opti-Clave 32oz
	Clean the internal (and external - if installed) water tank with Opticlave Autoclave Cleaner.				
	Inspect, clean, or replace the drain filter.				
	Ensure area around Bravo is dust free.				
Monthly	Test the safety valve located on the back of the unit.				 Bravo 21V Biological Filter  Bravo 21V Door Gasket
3 - 6 Months	Or every 500 Cycles replace the Biological Filter.				
Yearly	Replace the Door Gasket every 1000 cycles or 12 months (whichever is first).				
General	If printer paper is not printing properly, remove paper, tighten the paper around the roll and retry.				 Bravo 21V Thermal Paper  Bravo 21V QPM Kit
	Complete Bravo 21V PM Kit available includes: Door Gasket, Biological Filter, and Chamber Drain Filter				



Scican Bravo 21V

StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3



FREE TECH SUPPORT