









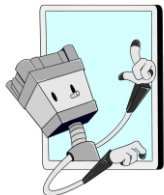


# Preventative Maintenance Checklist

	Completed by	Date	
<p>This document outlines the cleaning and preventive maintenance procedures needed to be maintained on your Hydrim C61W G4. <i>To see detailed instructions on each step please scan the QR code below titled "Hydrim C61W G4 Maintenance"</i></p>			 
<p><b>Replacing the HIP Cleaning Solution:</b> The HYDRIM C61W G4 uses a pre-measured container of HIP Cleaning Solution. Use only the recommended cleaning solution and read the MSDS before inserting into the unit.</p>			 
<p><b>Refilling the Water Softening Salt Reservoir:</b> When a red X appears on the detergent and water softener level indicator, press on the icon to determine if the problem lies with the water softener or cleaning solution level. If the water softener must be refilled follow the steps in the instructions.</p>			 
<p><b>Changing the Air Filter:</b> When the message "Replace air filter" appears, the HEPA filter is in need of changing. Follow the steps in the instructions.</p>			 
<p><b>Wipe Down Hydrim:</b> To clean outer surfaces and the decal covering the touchscreen, use a soft cloth moistened with soap and water or a surface disinfectant like OPTIM1 Disinfectant Cleaner. <i>Do not use harsh cleaning chemicals.</i></p>			
<p><b>Daily Maintenance:</b></p> <ul style="list-style-type: none"> <li>Wipe Down Outer Surfaces-</li> <li>Filter Maintenance-</li> <li>Wash Arm Maintenance-</li> <li>Door Lock Check-</li> <li>Wash Arm Rotation Check-</li> <li>Door Seal Check-</li> <li>Load Carrier Check-</li> <li>Chamber Filter Check-</li> </ul>  			
<p><b>Annually - By a SciCan Approved technician -</b> Annual maintenance schedule as outline in the service manual</p>			



## HYDRIM C61W G4

**StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3**



FREE TECH SUPPORT