










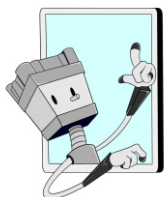


Preventative Maintenance Checklist

Frequency	Preventative Maintenance	Item	Completed by	Date	Order Link
Weekly	Log and monitor water quality readings. Notating the water quality readings for the Source, RO, D1, and D2 will expedite a solution if the need arises to contact Tech Support @ 704-966-1650 option 3				
As Needed	STAGE 4 CARTRIDGES Stage 4 Cartridge readings are based on the quality of water coming off them. Water quality readings can be found on the far right monitor. When the reading on the Water Quality Monitor degrades to 10ppm on D1, the audible alarm will sound, and the "Autoclave Water Quality" indicator lights on the front of the system and at the white Autoclave Faucet Disc, will illuminate red. At this point, the leading (left-most) Stage 4 will need to be changed.				 AC+4 Sterisil AC+ Stage 4 Cartridge  AC+DI Sterisil AC+ 3 Stage 4 Cartridges Stage 4 XL Cartridges Available Below
Yearly	Stages 1, 2, 3, and the UV Light's lifespan are chronological and are replaced annually from the date of installation. Their lifespan is tracked via the Filter Monitor on the far left side of the system. Check the pressure of the systems Water Storage Tank* *Also check any time a cartridge is changed!				 Sterisil AC+ Stage 1-3 Cartridges & Bulb AC+AK
	Other Cartridges/Parts Available	 AC+1 Sterisil AC+ Stage 1 Cartridge  AC+2 Sterisil AC+ Stage 2 Cartridge  AC+3 Sterisil AC+ Stage 3 Cartridge  AC+4XL Sterisil AC+ Stage 4 XL Cartridge  AC+DIXL Sterisil AC+ 3 Stage 4 XL Cartridges  SS-UV Sterisil AC+ Replacement UV Light Bulb			



Sterisil AC+

StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3



FREE TECH SUPPORT