









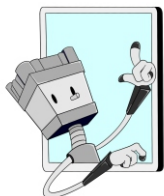


Preventative Maintenance Checklist

Frequency	Maintenance	Item	Completed by	Date	Order Link
Daily	Clean Exterior & Check Function Display 1 - Check the function display 2 - Clean the cover and the waste disposal tray. Check for unit cleanliness and clean as necessary. 3 - Disinfect surrounding area with Optim 1 Disinfectant	 OPTIM1 Disinfectant Wipes			 Optim 1 6"x7" Wipes
Weekly	Clean Exterior & Check Level Displays 1 - Clean the outside of the case with W&H Cleaning Liquid MC-1000 . 2 - Check the filling level displays. 3 - Disinfect surrounding area with Optim 1 Disinfectant.	 W&H Cleaning Liquid MC-1000			 W&H Cleaning Liquid MC-1000
Bi-Monthly or as needed	Check the O-rings on the maintenance attachment. 1 - Change the O-rings in the event of visible defects or every two months. 2 - Replace damaged or leaking O-rings IMMEDIATELY! 3 - Replace any missing O-rings IMMEDIATELY! Firmly squeeze the O-ring between thumb and forefinger and pull it off. Push on a new O-ring. The O-rings can be exchanged without the need for special tools. Grease-free fingers will facilitate this exchange.	 Assistina O-rings 02695700			 Assistina O-rings 02695700
7000 Cycles or as needed	Renewing the Filter Pack 1 - Renew the filter pack in the suction filter after approx. 7,000 maintenance cycles corresponds to the consumption of a 500 ml bottle of W&H service oil).	 Assistina Filter Pack 02670500			 Assistina Filter Pack 02670500
Yearly or as needed	Function & Filter Check 1 - Carry out a function check 2 - Check the fine filter in the air connection hose for contamination. Replace the air filter whenever it is dirty or at least 1x per year.	 Assistina Air Filter 02675200			 Assistina Air Filter 02675200



W&H Assistina 301 Plus

StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3



FREE TECH SUPPORT



Assistina 301 Plus Users Manual