















**Sterilizer Autoclave Solutions**  
INFECTION CONTROL SPECIALISTS

## Preventative Maintenance Checklist

| Frequency        | Preventative Maintenance  | Item   | Completed by | Date | Order Link  |
|------------------|---|--|--------------|------|---|
| Daily            | Replace distilled water in reservoir as needed  |   |              |      | <br>Optim1 6"x7" Wipes   |
|                  | Clean door gasket with a micro fiber cloth/dishwashing soap and inspect for crease or tears. Wipe exterior of unit and surrounding area with Optim disinfecting wipes |  |              |      |   |
| Weekly           | Clean the chamber, rack, and trays with T-Clean Chamber Shine (If you are not using this product, use Chamber Brite weekly instead of monthly)                        |    |              |      | <br>T-Clean Chamber Shine 500ml <br>Opti-Clave 32oz |
|                  | Remove chamber filter and rinse with water  |  |              |      |   |
|                  | Drain reservoir and use Opticlave to clean the water reservoir  |  |              |      |   |
|                  | Lubricate threads and hinge of door closing device with WD-40 or equivalent penetrating oil   |  |              |      |   |
| Monthly          | Run a Chamber Brite cycle to descale chamber and internal plumbing<br><br>For best results, run chamber brite every 20-25 cycles                                      |    |              |      | <br>Chamber Brite Powder   |
| Every Six Months | Replace door gasket and HEPA filter   |   |              |      | <br>EZ10K Door Gasket <br>EZ10K Hepa Filter       |
| Yearly           | Replace the safety valve, door gasket, air jet (black top), HEPA filter, and plungers   |  |              |      | <br>EZ10K PM Kit   |
|                  | *If you have questions please call our FREE Technical Support Line  |  |              |      |   |



## Tuttnauer EZ10K

**StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3**



FREE TECH SUPPORT