



Sterilizer Autoclave Solutions
INFECTION CONTROL SPECIALISTS

Preventative Maintenance Checklist

Frequency	Preventative Maintenance	Item	Completed by	Date	Order Link
Daily	Clean Door Gasket				 Optim1 6"x7" Wipes
	Empty the waste bottle every time you refill the reservoir				
	Run Bowie Dick test daily				
	Wipe surrounding area with Optim1 disinfecting wipes				
Weekly	Clean the internal part of the tray with a mild chlorine-free detergent. After cleaning rinse the tray thoroughly with water and dry before reinstalling.				 Enbio PRO Water In Tubing
	Clean the interior of the process chamber with a mild chlorine-free detergent. After cleaning wipe the chamber				
	Inspect the Front Seal, Hepa Filter, Drain Hose Connection, Container Plugs, and Enbio Magic Filter.				
	Clean Waste Water Bottle with OptiClave Cleaner				
After 100 Performed Cycles	Clean the seal. Use warm water and a microfiber cloth (microfiber with silver particles is acceptable). Do not use blunt/sharp cleaning tools or chemicals. Perform cleaning on a cooled device. Be careful not to bend the drawer. After cleaning, leave the device open until the seal dries.				 Enbio PRO Distilled Water Bottle
As needed by Time / Cycles / Damage	Quarterly - Check Water Tank for Contamination. If found empty, clean, and refill with fresh water.				 Enbio PRO Gasket
	6 Months or Amber Colored Resin - Replace Magic				
	1000 Cycles/365 Days - Hepa Filter Replacement				
	1500 Cycles/365 Days - Front Drawer w Chamber Seal				
	3000 Cycles/365 Days - Preventive Maintenance Service				
	Leaking / 365 Days - Connection / Drain Hose				
	Leaking - Water/Condensate Container Plugs				



Enbio PRO

StatimUSA.com - Free Technical Support Line 704-966-1650 Option 3



FREE TECH SUPPORT